
BILANCIA ASTRA
ASTRA SCALE
BALANCE ASTRA
WAAGE ASTRA
BÁSCULA ASTRA
BALANÇA ASTRA
ΖΥΓΑΡΙΑ ASTRA
Astra ميزان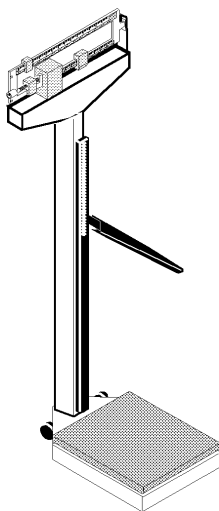

MANUALE D'USO E MANUTENZIONE
USE AND MAINTENANCE BOOK
INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN
BETRIEBS UND WARTUNGS ANWEISUNGEN
MANUAL DE USO Y MANTENIMIENTO
MANUAL DE USO E MANUTENÇÃO
ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ ΚΑΙ ΣΥΝΤΗΡΗΣΗΣ
دليل الإستعمال والرعاية

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FEATURES

The GIMA Astra scale is a device designed to measure and weigh people. The broad measuring range, high precision and sensitivity make it particularly suitable for hospitals, clinics, schools, family doctors' offices, gyms, etc.

Technical Specifications

Weight measurements

Maximum capacity: 200 Kg

Precision: 100 g

Non-type approved product class III

Height measurements

Measuring range: 75÷200 cm

Precision: 0,5 cm

Platform dimensions (L x W): 38,5 x 28 cm

Overall dimensions (L x W x H): 55 x 27,5 x 149,5 cm

Weight of scale: 21 Kg

Box contents

- Column and balance beam
- Platform
- Height measuring rod
- Wheels
- Screws and washers

PRESCRIPTIONS

The product shall be used by qualified medical personnel only.

Do not use the equipment in case it is damaged. Apply to your retailer.

Avoid precarious repairs. Repairs shall be carried out with original spare parts only, which shall be installed according to the intended use.

Since the product is made of corrosion-proof materials suitable for the environmental conditions foreseen for its normal use, does not require special care, however it is necessary to store it in a closed place making sure that is protected from dust and dirt to assure its hygienic conditions. Moreover, it is recommended to store the product in a place which can be reached easily by the personnel in case of necessity.

UNPACKING

Always remember that packing elements (paper, cellophane, stitches, adhesive tape, etc.) can cut and/or hurt if they are not carefully handled. They shall be removed with adequate means and shall not be left at the mercy of irresponsible persons; the same is valid for tools used to remove packages (scissors, knives, etc.).

After opening the packages, first of all it is necessary to check all pieces and parts composing the product. Check that they are all present and in perfect conditions.

INSTALLATION

The device is mostly factory pre-assembled, therefore assembly is very easy (see illustration):

- Fix the column to the base using the supplied screws and washers
- Lay the scale on the floor and hook the transmission rod onto its seat
- Fix the wheels to the base with the supplied screws and washers
- Fix the measuring ruler to the two hexagonal-head screws pre-assembled on the column.
- Fix the ruler by tightening the hexagonal-head screws using the relative wrench. Don't tighten too much!

To calibrate the scale, after setting both weights to zero, work on the adjustment screws (see illustration) until the indicator matches the measurement tab.

The base must be positioned on a perfectly flat floor to calibrate the unit correctly.

OPERATION

Before the person steps onto the platform, bring the height cursor up above the person's height. After the person is on the platform, slowly lower the cursor until it rests on the person's head.

The graduated measuring beam can be extended on both sides: the lower part can be used to measure heights up to 102 cm. The height value is indicated right by the cursor. The upper ruler is used to measure heights up to 200 cm. The height value is indicated right where the two graduated measuring beams intersect, indicated by the READ arrow (see illustration).

Note: to correctly use the upper measuring beam block the cursor on the upper part of the sliding measuring ruler.

Attention: Do not keep the cursor horizontally because it may be dangerous.

MAINTENANCE

The device was designed to last a lifetime. It is resistant to chemical attack, except for M.E.K. ketons, diluting solutions and polish removers.

Periodically verify the calibration and the leveling of the scale. Perform this operation every time it is moved (see installation).

GIMA WARRANTY CONDITIONS

Congratulations for purchasing a GIMA product.

This product meets high qualitative standards both as regards the material and the production.

The warranty is valid for 12 months from the date of supply of GIMA.

During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included.

All components subject to wear are not included in the warranty.

The repair or replacement performed during the warranty period shall not extend the warranty.

The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use.

GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electro-magnetic fields, radio interferences, etc.

The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed.

The defected products must be returned only to the dealer the product was purchased from.

Products sent to GIMA will be rejected.

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