# GIMA

**BARELLA TRENDELENBURG**  
**TRENDELENBURG PATIENT TROLLEY**  
**CIVIERE TRENDELENBURG**  
**BETT TRENDELENBURG**  
**CAMILLA TRENDELENBURG**  
**MACA TRENDELENBURG**  
**ΦΟΡΕΙΟ TRENDELENBURG**  
**TRENDELENBURG نقالة مرضى**

**MANUALE D'USO E MANUTENZIONE**  
**USE AND MAINTENANCE BOOK**  
**INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN**  
**BETRIEBS UND WARTUNGSANWEISUNGEN**  
**MANUAL DE USO Y MANTENIMIENTO**  
**MANUAL DE USO E MANUTENÇÃO**  
**ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ ΚΑΙ ΣΥΝΤΗΡΗΣΗΣ**  
**دليل الإستعمال والرعاية**

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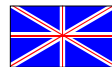
## FEATURES

The trolley's intended use is to facilitate the transport with a suitable accommodation of the patient's posture (Trendelenburg up to 9° - anti-Trendelenburg up to 14°). The patient's safety is also guaranteed by a bumper **A** and tilting side boards **B**.

The trolley has been conceived to be used indoor as an aid for patients in nursing homes. The max. carrying capacity is 100 Kg. calculated with the rack completely in horizontal position.

The product, or parts of it, cannot be used for uses different from the one specified in the present manual.

## PRESCRIPTIONS

Place the patient carefully on the bed only after providing for a suitable mattress and with the bedspring completely in horizontal position everywhere.

Do not use stiff mattresses (it is advisable to use mattresses suitable for articulated beds)

The trolley must be used on even surfaces and/or with little inclination. In any case always use the parking brakes **E**.

*N.B. The head board **H** and tilting sideboards **C** areas, as well as the frame under the rack are exposed to crushing risks; be very careful.*

*Do not use the equipment in case it is damaged. Apply to your retailer. Avoid precarious repairs. Repairs shall be carried out with original spare parts only, which shall be installed according to the intended use.*

Since the product is made of corrosion-proof materials suitable for the environmental conditions foreseen for its normal use, does not require special care, however it is necessary to store it in a closed place making sure that is protected from dust and dirt to assure its hygienic conditions. Moreover, it is recommended to store the product in a place which can be reached easily by the personnel in case of necessity.

## UNPACKING

*Always remember that packing elements (paper, cellophane, stitches, adhesive tape, etc.) can cut and/or hurt if they are not carefully handled.*

*They shall be removed with adequate means and shall not be left at the mercy of irresponsible persons; the same is valid for tools used to remove packages (scissors, knives, etc.).*

After opening the packages, first of all it is necessary to check all pieces and parts composing the product. Check that they are all present and in perfect conditions.

## INSTALLATION

*During the assembly do not lean the bedsprings or the bedheads against the walls! They may fall down or slip and cause damage to people and things.*

*Moreover pay attention to hinged elements, which can crushed your fingers.*

*Attention to burrs on pipes and couplings, if present remove them with suitable tools.*

*They can cause damage during the assembly or the use of the product.*

The two wheels without brakes are mounted on the headboard and the braking ones **E** are installed on the footboard.

During assembly the wheels' brakes **E** must be blocked.

Once the installation has been completed, check for the correct tightening of the nuts locking the legs and for the correct insertion of the headboard and the footboard.

The assembly consists in simply installing the headboard and the footboard with relevant bumpers A as shown in the pictures. The I.V. stand **B** and the tilting sideboards **C** are supplied on request.

### OPERATION

To lift the trolley, press the pedal **F** without pushing it to the travel end.

To lower the trolley, press the pedal **F** until reaching the travel end.

When the trolley is lifted it is possible to lift the headboard **H** by means of the adjacent levers.

The trolley is provided with the Trendelenburg function up to 9° and anti-Trendelenburg up to 14°. To reach such inclinations, make use of the lever **G** and press or lift the rack until the desired position is reached, then release the lever **G**.

The tilting sideboards **C** are hinged. For this reason after they have been lifted they remain locked in this position due to gravity; to lower them it is necessary to pull them upwards and tilt them paying attention not to let make them fall.

Usually the handwheels **D** must be left unscrewed or open; they can be used to permanently block the position of the trendelenburg device in case of emergency when the system of inclination by means of lever **G** does not work.

### MAINTENANCE

Clean the bed without using abrasive detergents or solvents (it is advisable to use water and soap). After washing, carefully dry each part of the bed.

Periodically check whether the bedspring, hinges and articulations are worn out and in case of necessity provide for repairs or replacement.

Any extraordinary maintenance operation must be carried out exclusively by authorized personnel only.

### GUARANTEE

Thank you for having acquired our product. This product meets the most stringent requirements regarding the selection of quality manufacturing materials and also the final control. The product has a 12-month warranty, valid from the date it is delivered by GIMA.

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
During the guarantee period free repair and/or replacement of any defective parts due to faulty manufacture will be given, labour, postal, transport and packaging charges etc. are not included. The guarantee therefore excludes components subject to wear and tear such as parts in rubber or PVC, etc. No compensation will be given for loss of use of the product.

Furthermore replacement or repairs effected during the guarantee period do not lengthen the duration of the guarantee.

*This guarantee is void in the event of: repairs effected by unauthorised persons or with spare parts not approved by GIMA, damage or defects caused by negligence, blows, abnormal use of the appliance or faulty installation.*

The guarantee will be invalidated if the registration number has been removed, cancelled or altered. Faulty appliances must be returned only to the retailer where the item was bought. Any item dispatched directly to us will be refused.

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