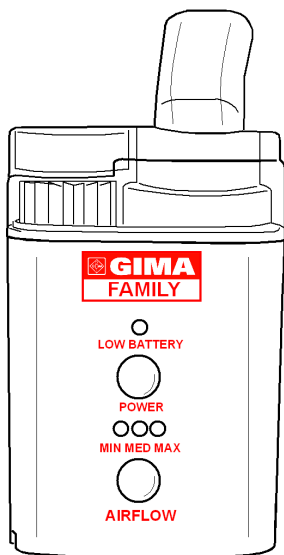
AEROSOL AD ULTRASUONI FAMILY
FAMILY ULTRASOUND AEROSOL
GÉNÉRATEUR D'AÉROSOL À ULTRASONS FAMILY
ULTRASCHALL-AEROSOL FAMILY
NEBULIZADOR ULTRASÓNICO FAMILY
AEROSOL DE ULTRA-SONS FAMILY
ΤΟ ΑΕΡΟΣΟΛ ΜΕ ΥΠΕΡΗΧΟΥΣ FAMILY

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FEATURES

The Family Ultrasound Aerosol is ideal for the treatment of asthma, of respiratory problems and other pathologies of the breathing organs, which require aerosol drug therapy. It operates with running or distilled water and the aerosol drug prescribed by the doctor. The product cannot be used with Pentamidine or other substances that are not suitable for inhalation.

Technical Features

Power supply: 230V AC 50Hz - 10W

Ultrasound frequency: 2,5 MHz

Nebulization: from 0.2 to 0.7 ml/minute according to the density of the medicine

Capacity of the medicine holder cup: maximum 8 ml

Timer: approx. 10 minutes

Dimensions (without battery): 8,2 cm x 5 cm x 13,2 cm

Weight (without battery): approx. 250 gr.

Accessories: Power adapter **R** , case, mouth piece **A** , 5 re-usable medicine holder cups

E, Masks **S** , extensible tube **T**, travel power adapter and rechargeable battery **Q**

Type of battery: Ni-MH 12V DC - 600mAH

CAUTION

- The supervision of an adult is necessary when the unit is used with children; do not leave the unit, even when it is switched off, or any of its components, near the reach of children.

- Do not expose eyes to the steam coming out of the unit

- Follow the aerosol therapies only upon doctor's suggestion. Do not nebulize incorrect substances or substances that are not authorized by your doctor!

- DO NOT use the unit near water sources (sink, bath tub, etc.) or near other electric appliances.
- Do not use the unit if it is damaged. Contact your dealer and avoid any temporary repair.
- Do not block the air inlet from the ring **B** with towels or other objects.
- Do not open the lid **D** during use.
- Disconnect the device from the mains before cleaning it, filling it and in any case at the end of each use.
- Empty out the water from the tank **I** after each use. Only water can be used inside the tank. Other substances may damage the product!

OPERATION

*Caution: Do not switch on the device if the tank **I** or medicine holder cup **E** are empty. Always fill the tank **I** with water up to the level **H** between the two marks. ALWAYS use the medicine holder cup **E**. The water in the tank **I** is used to transfer the ultrasounds and cool off the device and NOT to dilute the drug!*

Remove the lid **D** by pulling it upwards. Remove the medicine holder cup **E** and pour running water or distilled water (preferable) at room temperature (20/30°C) inside the tank **I**. Fill the tank **I** up to the level **H** indicated between the two marks.

Fix the medicine holder cup **E** above the tank and pour over the medicine. The maximum capacity of the cup is 8 ml, the ideal quantity is 4 ml. If necessary (for dense drugs with less than 2 ml of volume) dilute the drug with distilled water or saline solution (follow the doctor's instructions).

Close the lid **D** until it clicks.

Connect the power adapter **R** to the power outlet and the plug to the outlet under the unit. When using the rechargeable battery **Q** assemble as described below.

Put the unit on a stable surface and assemble the mouth piece according to need **A**, the flexible tube **T** and/or the mask **S**.

Press the ON/OFF button **M**. The unit is set to start operating at the maximum air flow rate. The air flow can be adjusted using the appropriate button **O**. Indicator **N** shows the current level. The device has 3 flow options MAX (maximum), MED (medium), MIN (minimum).

Use the flow suggested by the doctor and keep in mind that the nebulization increases gradually during the application. The channeling cone **G** increases the nebulization power for the thicker drugs if necessary.

Switch off the unit and disconnect it after the therapy. The unit is fitted with an automatic safety switch-off function after ten minutes of continuous use.

NOTE: Wait 30 minutes between one therapy cycle and the next.

After each use:

- Remove the power adapter **R** from the outlet
- Let the unit cool off (at least 30 minutes)
- Remove the lid **D**
- Remove and empty out the medicine holder cup **E**
- Empty out the tank **I** and dry it with a soft cloth
- If you don't need to use the unit again, follow the MAINTENANCE operations.

Using the rechargeable battery

To connect the battery to the unit: position the unit **P** on the battery **Q** and let the two elements slide and click into place; to remove them have them slide in the opposite direction.

Warning: Do not expose the battery to heat or humidity. Avoid placing the battery on metallic surfaces. There is risk of short circuit or bursting.

To recharge the battery use the same power adapter, and insert the adapter's plug in the outlet under the battery.

For the first use or after a long period of inactivity, let the battery charge for 24 hours. If the unit is used regularly 4/12 hours are enough. The fully charged battery has an autonomy of 20/30 minutes of continuous use, to maintain the efficiency of the battery we suggest completely charging it after every use.

MAINTENANCE

Disconnect the power adapter **R** of the unit from the mains and let it cool off at least 30 minutes before cleaning it. Clean the unit after each use. The main body of the unit **MUST NOT** be immersed into liquids. Simply clean it with a damp cloth.

Components such as: the mouth piece **A**, the masks **S**, the flexible tube **T**, the lid **D**, the medicine holder cup **E**, the ring **B**, the fan **F**, and the air filter **C** can be cleaned with a mild detergent or disinfectants, rinse with water and dry before reassembling.

The unit has 5 spare medicine holder cups **E**. For any other extra components contact your dealer. The rechargeable battery **Q** has a duration of 2 years according to the frequency and modality of use.

If the unit does not switch on after pressing the ON/OFF button:

- Verify that the power adapter was correctly connected **R**
- Verify that the tank **I** was filled up and that the water is at the right level **H**
- If you are using the rechargeable battery **Q** it could be exhausted or not connected properly.

If the unit does not nebulize or does not nebulize enough:

- Verify that the medicine holder cup **E** has enough medicine in it
- Verify that the medicine holder cup **E** is not broken
- Verify (when the unit is cold and disconnected) that inside the tank **I** there is no scale build-up.

If the fan F does not operate correctly:

- Verify that it is correctly fixed and that there is nothing blocking it.

Disposal: *The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.*

For further information on recycling points contact the local authorities, the local recycling center or the shop where the product was purchased. If the equipment is not disposed of correctly, fines or penalties may be applied in accordance with the national legislation and regulations.

GIMA WARRANTY CONDITIONS

Congratulations for purchasing a GIMA product.

This product meets high qualitative standards both as regards the material and the production. The warranty is valid for 12 months from the date of supply of GIMA.

During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included.

All components subject to wear are not included in the warranty.

The repair or replacement performed during the warranty period shall not extend the warranty.

The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use.

GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electro-magnetic fields, radio interferences, etc.

The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed.

The defected products must be returned only to the dealer the product was purchased from. Products sent to GIMA will be rejected.

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