# GIMA

**SPATOLE DI AYRE**  
*AYRE SPATULE*  
**SPATULES DE AYRE**  
*AYRE SPATEL*  
**ESPÁTULAS DE AYRE**  
*ESPÁTULA DE AYRE*  
**ΣΠΑΤΟΥΛΑ ΤΟΥ ΑΥΡΕ**  
*AYRE مسواط*

**MANUALE D'USO E MANUTENZIONE**  
*USE AND MAINTENANCE BOOK*  
**INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN**  
*BETRIEBS UND WARTUNGS ANWEISUNGEN*  
**MANUAL DE USO Y MANTENIMIENTO**  
*MANUAL DE USO E MANUTENÇÃO*  
**ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ ΚΑΙ ΣΥΝΤΗΡΗΣΗΣ**  
*دليل الإستعمال والرعاية*

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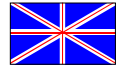
## AYRE SPATULE

The Ayre Spatula GIMA is an extremely efficient device for esocervical and endocervical cytological sampling (Pap Test).

The device must be used only by qualified trained personnel authorized to carry out the pap test.

Do not expose to sources of heat. Use only if the packing is not damaged. Extract the device from the package immediately before use. Before use, visually examine the device to check its integrity. Carefully check whether the wood is chipped.

**Intended purpose:** The spatula is used to take samples of cells exfoliated from the esocervix and from the endocervical channel.

**Contraindications:** Do not use extremity **1** in case of pregnancy.

**Instructions for use:** insert the speculum into the vagina to point out the cervix uteri; take the spatula from its packing holding it in the centre of the stem; visually check for the device integrity. Slowly introduce the extremity **1** into the cervical channel; carry out a complete rotation; slowly take it out; smear both spatula surfaces on the glass slide.

In case the cervical channel is stenotic or shows a pregnancy condition, make use of extremity **2**, by making it adhere to the esocervical surface and proceed as above-mentioned.

*The device IS NOT STERILE!*

*The device is SINGLE-USE. Do not clean for a subsequent use!*

The product must be disposed according to the local provisions on dangerous medicines disposal.

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