GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italy
Italia: tel. 199 400 401 - fax 199 400 403
Export: tel. +39 02 953854209/221/225 fax +39 02 95380056
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

Bracciale per trasfusione
Infusion Cuff Set
Manchette de perfusion à pression
Manschette für Transfusion
Brazalete de transfusión
Braçadeira para transfusão
Περιβραχιόνιο για μετάγγιση

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Infusion Cuff Set

This device is applicable to be used as the ancillary of infusion in clinic, and it is recommended to be used together with the infusion bag. Comparing with the traditional infusion method with pressure provided by gravity, this device can avoid the contamination of the ambient air as no ambient air will contact with the liquid medicine. In the meantime, it makes the paramedic more convenient as the paramedic don't need to hold the infusion set at given level accompanying the patient, especially when the patients go for eating, outdoor activity or toilet.

The device consists of infusion cuff, inflation bulb, manometer and knot. Please connect these parts before use.

The instruction of using this instrument

This device can provide necessary pressure for the infusion bag so as to ensure the liquid medicine flows from the infusion bag to the patients' body.

- Put the infusion bag into the infusion cuff, and the infusion bag will be just below the transparent net. Put the infusion cuff in safe area to avoid extrusion or falling down, fix the infusion bag by the pressure of infusion cuff;

- Pump the inflation bulb to inflate air into the inflation cuff. Observing the manometer carefully to make sure that the pressure reaches the average of the patient's systolic pressure and diastolic pressure;

- Connect the disposable infusion set with the infusion bag, and the liquid medicine will flow into the whole disposable infusion set. Make sure that there is not bubble in it, and close the infusion switch. Then it is ready for other ordinary infusion operation;

- During the infusion, the pressure will decrease gradually. Please pump the inflation bulb to increase the pressure. If the infusion speed is too low or too high, please modify the pressure in the infusion cuff by pumping the bulb to inflate or releasing deflation valve to deflate so as to get expected infusion speed;

GIMA WARRANTY CONDITIONS

Congratulations for purchasing a GIMA product.

This product meets high qualitative standards both as regards the material and the production.

The warranty is valid for 12 months from the date of supply of GIMA.

During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included.

All components subject to wear are not included in the warranty.

The repair or replacement performed during the warranty period shall not extend the warranty.

The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use.

GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electro-magnetic fields, radio interferences, etc.

The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed.

The defected products must be returned only to the dealer the product was purchased from.

Products sent to GIMA will be rejected.

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