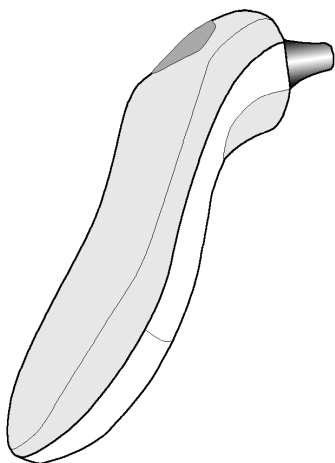
**TERMOMETRO AURICOLARE
IR EAR THERMOMETER
THERMOMETRE AURICOLAIRE
OHRENTHERMOMETER
TERMÓMETRO AURICULAR
TERMÔMETRO AURICULAR
ΘΕΡΜΟΜΕΤΡΟ ΑΥΤΙΟΥ**

**MANUALE D'USO E MANUTENZIONE
USE AND MAINTENANCE BOOK
INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN
BETRIEBS UND WARTUNGS ANWEISUNGEN
MANUAL DE USO Y MANTENIMIENTO
MANUAL DE USO E MANUTENÇÃO
ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ ΚΑΙ ΣΥΝΤΗΡΗΣΗΣ**

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PREMISE

Thank you for purchasing this GIMA Ear-type thermometer. This thermometer has been planned with advanced infrared technologies and room environment compensation, for accurate and prompt measurements. Accuracy is $\pm 0.2^{\circ}\text{C}$ (0.4°F) during $35.5\text{--}42^{\circ}\text{C}$ ($95.9\text{--}107.6^{\circ}\text{F}$), $\pm 0.3^{\circ}\text{C}$ (0.5°F) for other range. (IEC60601-1)

Do not use this device for Live & Die decision or Safety related applications.

FUNCTIONING

Before taking any measurement, make sure that the ear canal is clean, dry and not obstructed.

1) Pull off the probe cap **(1)**. Do not force in removing the cap.

2) It is necessary to apply a disposable plastic cap **(2)** on the probe **(3)** by means of the apposite connection ring **(4)**:

- Place the plastic cap **(2)** on the connection ring **(4)** in conformity with the direction indicated in the picture;
- Push the ring on the probe until a click sounds;
- Once the cap has been fixed, delicately remove the connection ring **(4)**.

Always make use of a new disposable plastic cap and make sure that it has not been damaged.

3) Press ON/MEM **(6)** pushbutton. When the ear icon stops flashing and the equipment beeps twice, the thermometer is ready for use.

4) Gently pull the ear back to straighten the ear canal and snugly position the probe **(3)** in the ear canal, aiming towards the membrane of the eardrum to obtain an accurate reading.

5) Measuring the ear temperature:

Hold the SCAN **(5)** pushbutton pressed with your forefinger until you hear a beep. At this point the LCD display **(7)** shows the detected temperature.

6) Device will automatically power off if left idle for more than 60 seconds to extend battery life.

SWITCHING BETWEEN FAHRENHEIT AND CELSIUS

To change the unit of measurement of temperature from $^{\circ}\text{C}$ to $^{\circ}\text{F}$, hold down the SCAN button **(5)**, when the unit is off, then press and hold the ON/MEM button **(6)** pressed until the display shows the symbol $^{\circ}\text{F}$ or $^{\circ}\text{C}$. Then release both buttons. Use the same process to change the unit of measurement again.

MEMORY FUNCTION

If the reading thermometer is within the normal temperature range (34°C ~ 42.2°C or 93.2°F ~ 108°F), the data is saved into memory each time the thermometer is turned off.

Press the ON/MEM button **(6)** when the thermometer is turned on to see the temperature stored.

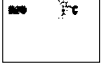
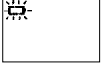
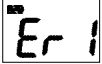
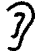
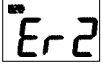
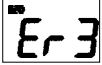
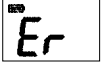
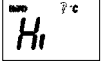
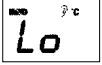
WARNING

- Follow the instructions of the manual to the letter for the best measurement.
- It is advisable to stay in the same environment for at least 5 minutes before taking the temperature.
- Do not have a bath or physical exercise before a measurement.
- While measuring avoid exposing to direct sunlight, rain or wind; do not take measurements near source of heat or very humid places.
- Keep far from children's reach both the connection ring (4) and the disposable plastic caps (2).

BATTERY REPLACEMENT

This device is supplied with one lithium cell (CR2032 x 1). Remember to power off the unit before replacing the battery. A malfunction may occur if the power is on when the battery is replaced. If a malfunction occurs, restart the device. Keep the battery away from children.

PROBLEM SOLVING

Error message	Problem	Solution
	Device stabilization in process.	Wait until  stops flashing.
	Battery is low and no more measurements are possible.	Replace the battery.
	Measurement before device stabilization.	Wait until  stops flashing.
	The device showing a rapid ambient temperature change.	Allow the thermometer to rest in a room at least 30 minutes at room temp. (10°C ~ 40°C, 50°F ~ 104°F).
	The ambient temperature is not within the range between 10°C and 40°C (50°F ~ 104°F).	Allow the thermometer to rest in a room at least 30 minutes at room temp. (10°C ~ 40°C, 50°F ~ 104°F).
	Error 5 ~ 9, the system is not functioning properly.	Unload the battery, wait for 1 minute and repower it. If the message reappears, contact the retailer for service.
	Temperature taken is higher than 42.2°C (108°F).	Check the integrity of the probe cover and take a new temperature measurement.
	Temperature taken is lower than 34°C (93.2°F).	Make sure the probe cover and the lens are clean and take a new temperature measurement.
	Device can not be powered on to the ready stage.	Change with a new battery.

CLEANING AND STORAGE

Replace the disposable plastic cap after each use to ensure an accurate reading and avoid cross infections.

In case multiple consecutive measurements are taken on the same patient, clean the probe plastic cap (2) with a soft cloth moistened with medical alcohol.

The probe lens (8) is the most delicate part of the thermometer. Use with care during cleaning to avoid damage. If the device is accidentally used without a plastic cap (2), clean the probe lens as:

- a) Gently wipe lens surface with a cotton swab or soft cloth moistened with water or medical alcohol;
- b) Allow the lens to fully dry at least 30 minutes before attaching a new probe cap.

Always store at a temperature between 10°C to 40°C (50°F ~ 104°F), in a dry place and far from liquids and direct sunlight. In case extraordinary maintenance is required, apply to the closest retailer.

***Disposal:** The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment. For further information on recycling points contact the local authorities, the local recycling center or the shop where the product was purchased. If the equipment is not disposed of correctly, fines or penalties may be applied in accordance with the national legislation and regulations.*

GIMA WARRANTY CONDITIONS

Congratulations for purchasing a GIMA product. This product meets high qualitative standards both as regards the material and the production. The warranty is valid for 12 months from the date of supply of GIMA. During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included. All components subject to wear are not included in the warranty. The repair or replacement performed during the warranty period shall not extend the warranty. The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use. GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electro-magnetic fields, radio interferences, etc. The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed. The defected products must be returned only to the dealer the product was purchased from. Products sent to GIMA will be rejected.

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