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**Disposal:** The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment. For further information on recycling points contact the local authorities, the local recycling center or the shop where the product was purchased. If the equipment is not disposed of correctly, fines or penalties may be applied in accordance with the national legislation and regulations.

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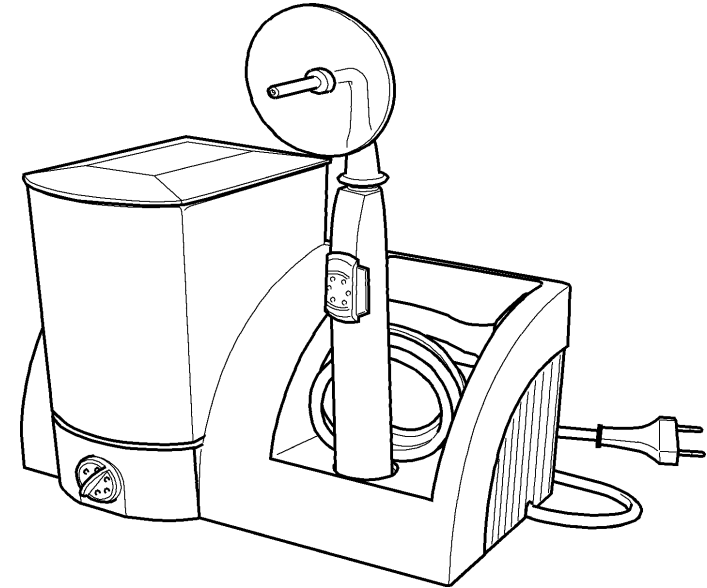
**ATTENTION:** The operators must carefully read and completely understand the present manual before using the product.

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## OTOSCILLO USE AND MAINTENANCE BOOK

### FEATURES

The irrigation device generates a pulsating water jet with adjustable pressure. It is equipped with a nozzle with a little guard sheet **C**, suitable for otologists. The device may only be used in medical practices and only as irrigation device for the auditory canal. The product or its components cannot be used for usages different to the ones specified in the present manual.

### PRESCRIPTIONS

*Plug in device only into a properly installed socket. The device has an alternating voltage of 220 volts.*

*Fix device on the wall or put it on a surface in a way that it cannot fall into water. Do not dip it into water! Before cleaning always pull out plug from the socket.*

Use device only with a full water container **A**.

Empty water container **A** totally after use.

Operating time up to a maximum of 12 minutes.

Use exclusively water for rinsing. For cleaning and disinfection of the appliance decalcifying and disinfecting agents can be used provided they are properly measured.

Flex has to be examined regularly for damages.

*Do not use device if damages are detected. Go to an expert electrician.*

*Avoid precarious repairs. Repairs shall be carried out with original spare parts only, which shall be installed according to the intended use.*

### UNPACKING

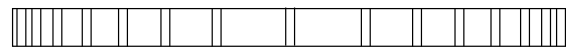
*Always remember that packing elements (paper, cellophane, stitches, adhesive tape, etc.) can cut and/or hurt if they are not carefully handled.*

*They shall be removed with adequate means and shall not be left at the mercy of irresponsible persons; the same is valid for tools used to remove packages (scissors, knives, etc.).*

After opening the packages, first of all it is necessary to check all pieces and parts composing the product. Check that they are all present and in perfect conditions.

### INSTALLATION

Apart from the use as a table device it can be fixed detachably to the wall using the given fixing system **F**. Because of automatic emptying of the system the installation height has to be chosen in a way that, while in use, the handle **D** is clearly over the level of the device!

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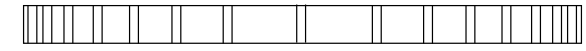
### 1. Filling water

Remove the water basin **A** pulling it upwards and fill it by WARM WATER, only after having checked the temperature (see running instructions).

Insert the filled basin **A**, placing it well on its base carefully.

### 2. Putting on the nozzle

Put square jet on the handle **D** and press it until it engages. Put on guard sheet **C**.

### 3. Quick filling of the tube system

Lowering the handle **D** under device level for a short time - before switching it on **E** the first time - helps the quick filling of the tube system. Put handle **D** always back into the handle mounting of the device to avoid automatic emptying of the water container and the tube system.

**WATCH OUT!** Do not fix over washbasin etc.!

## FUNCTIONING

### 1. Anamnesis and otoscopy

Otoscopy and questioning of the patient are necessary before the irrigation process.

In case of a tympanic-membrane perforation and/or an inflammatory process in the auditory canal an irrigation is not advised.

In case of bad anamnesis the patient should be asked whether he/she has ever suffered from otorrhea, whether water in the auditory canal has caused pains or whether former irrigations of the auditory canal have already caused problems.

### 2. Temperature of water

**IMPORTANT!** To avoid irritations the irrigation of the auditory canal has to be carried out with warm water! water temperature has to be checked immediately before the irrigation process!

To facilitate temperature checking, the device is supplied with the provided Temp.Test-Card.

Before filling, place the card under the water spurt. When temperature is correct (38° C) the card indicator is green. Should it remain green for 2-3 seconds, the container can be filled. The lower levels are signalled by blue colour while the higher levels are in brown colour.

After having filled the container, the card is to be hung on the back wall of the **A** basin, in order that the indicator is under the water level. That way water temperature can be monitored anytime, which is particularly useful if washing is not immediately carried out.

### 3. Positioning of the irrigation jet

**IMPORTANT!** The irrigation jet has always to be directed towards the meatus walls! Do never direct the irrigation jet towards the tympanic membrane!  
Do never position the irrigation nozzle far into the auditory canal. Put it at the beginning of the auditory canal towards the meatus walls.  
The nozzle position must always grant the outlet of rinsing water!

### 4. Adjusting the pressure

A general adjustment of the pressure **B** valid for every irrigation cannot be given as it depends, among other things, on the prevailing situation in the auditory canal, the sensitivity of the patient and not least the irrigation technique of the doctor.

**IMPORTANT!** Therefore it is absolutely necessary to get an impression of the range of the pressure by means of the finger pulp test.  
Put the irrigation jet towards the finger pulp of the middle finger with different pressure levels.

In view of the higher sensitivity of the auditory canal with this test you get a good idea of how to adjust the pressure **B**.

The test is to be carried out always before each washing, because in case of bad maintenance, the limestone deposits in the tube system can increase the stream pressure.  
For an ear canal cleaning on an adult subject a 2 level for the test washing is suggested while for the chief washing a 3-4 level is recommended.

*Pressure levels 5-6 are foreseen only for maintenance process!*

### 5. Irrigation test

A short irrigation test is carried out with an almost pressureless irrigation jet. So the sensitivity of the patient can be tested and at the same time he/she gets used to the irrigation.

Reactions of the patient can indicate e. g. a tympanic membrane perforation which could not be seen by means of the otoscopy!

### 6. Irrigation

The pulsation of the irrigation jet has the effect of a cerumen detachment even with a relatively low pressure of the irrigation jet.

Usually the washing process is carried out with a 3-4 level pressure, lasting for no more than 15 seconds.

*Do not raise the pressure while washing!  
If the patient does not feel well stop washing immediately!*

Soft to medium-hard cerumen can mostly be removed without local treatment but simply putting emollient drops into ears. In case of a complete cerumen obstruction a small hole should be scratched into the cerumen with a little hook.

Observation of the collected irrigation water can indicate the success of the irrigation.

After a cerumen detachment the auditory canal should be dried and disinfected.

### MAINTENANCE

Wipe container **A**, handle **D** and nozzle only with a damp cloth and dry them afterwards. Pull out plug! The **A** basin and the tube system should be disinfected regularly. Fill the **A** basin in (volume about 600 ml.) with water, add a disinfectant for appliances and let act as long as necessary. Wash the tube system with the nozzle. Pump the water with the disinfectant away and rinse with clear water with 5-6 level pressure out, repeatedly.

The nozzles can be disinfected by a cold disinfectant used for appliances.

Decalcification can be carried out like disinfection using the proper additive, otherwise pure wine vinegar can be used.

### GUARANTEE

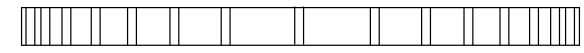
Thank you for having acquired our product.

This product meets the most stringent requirements regarding the selection of quality manufacturing materials and also the final control. The product has a 12-month warranty, valid from the date it is delivered by GIMA.

During the guarantee period free repair and/or replacement of any defective parts due to faulty manufacture will be given, labour, postal, transport and packaging charges etc. are not included. The guarantee therefore excludes components subject to wear and tear such as parts in rubber or PVC, doppler probes, SpO<sub>2</sub> sensors, lamps, batteries, electrodes, handpieces, washers and packings, resistances, etc.

No compensation will be given for loss of use of the product.

Furthermore replacement or repairs effected during the guarantee period do not lengthen the duration of the guarantee.

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*This guarantee is void in the event of: repairs effected by unauthorised persons or with spare parts not approved by GIMA, damage or defects caused by negligence, blows, abnormal use of the appliance or faulty installation.*

The guarantee will be invalidated if the registration number has been removed, cancelled or altered. Faulty appliances must be returned only to the retailer where the item was bought. Any item dispatched directly to us will be refused.

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