GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italy

Italia: tel. 199 400 401 - fax 199 400 403

Export: tel. +39 02 953854209/221/225 - fax +39 02 95380056

gima@gimaitaly.com - export@gimaitaly.com

www.gimaitaly.com

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I. Safety notes

Within this user manual are used these two symbols:

Information about hazards causing death, heavy physical injuries and psychological diseases or heavy material damage.

Information about hazards causing low physical injuries and psychological diseases or material damage.

Make absolutely sure that the instrument is operated at the main voltage indicated on the type plate at the bottom of the light source unit!

Always disconnect power plug before pulling out the lamp holder or opening the light source!

Repairs not described in this document must be carried out only by qualified workshops!

This instrument has not been approved for operation in areas subject to explosion hazards!


Before power on the instrument, check if the electrical system is earthed!

The manufacturer is not liable for any damage caused by the non-compliance with the above hints!

This instrument was designed and build according to the following rules and standards:

- CE 93/42 D.L. n° 46 del 24/02/97.
- CEI EN 60601-1
- CEI EN 60601-1-2

II. Description

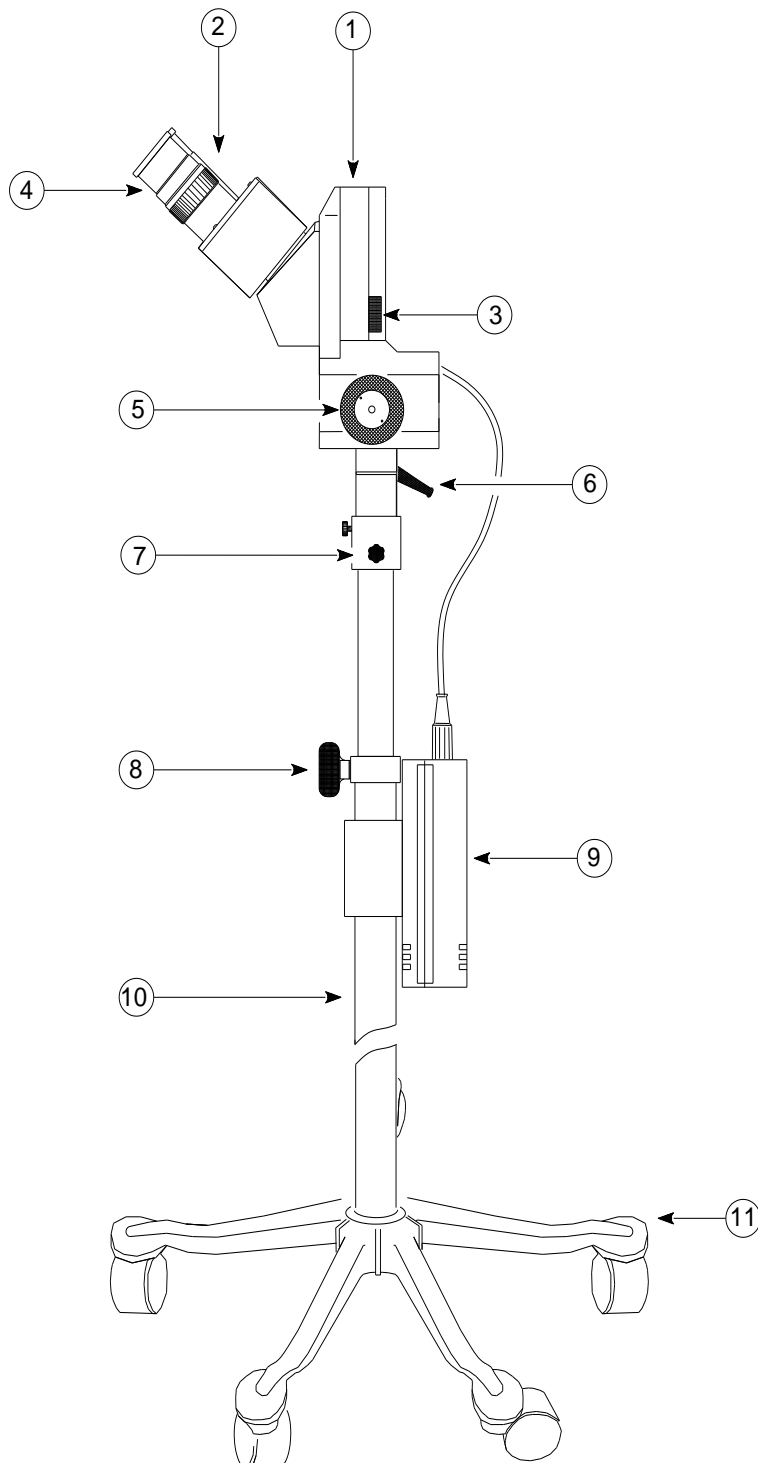- 12. Optical body
- 13. Diopter compensation
- 14. Green filter
- 15. Oculars
- 16. Focusing knob
- 17. Tilt tension adjustment knob
- 18. Pan tension adjustment knob
- 19. Height adjustment knob
- 20. Power unit
- 21. Stand
- 22. Base

fig. 1

III. Assembly

Before loosening the knob (8) (Fig. 1) assemble the optic head and the stand (see point 3).


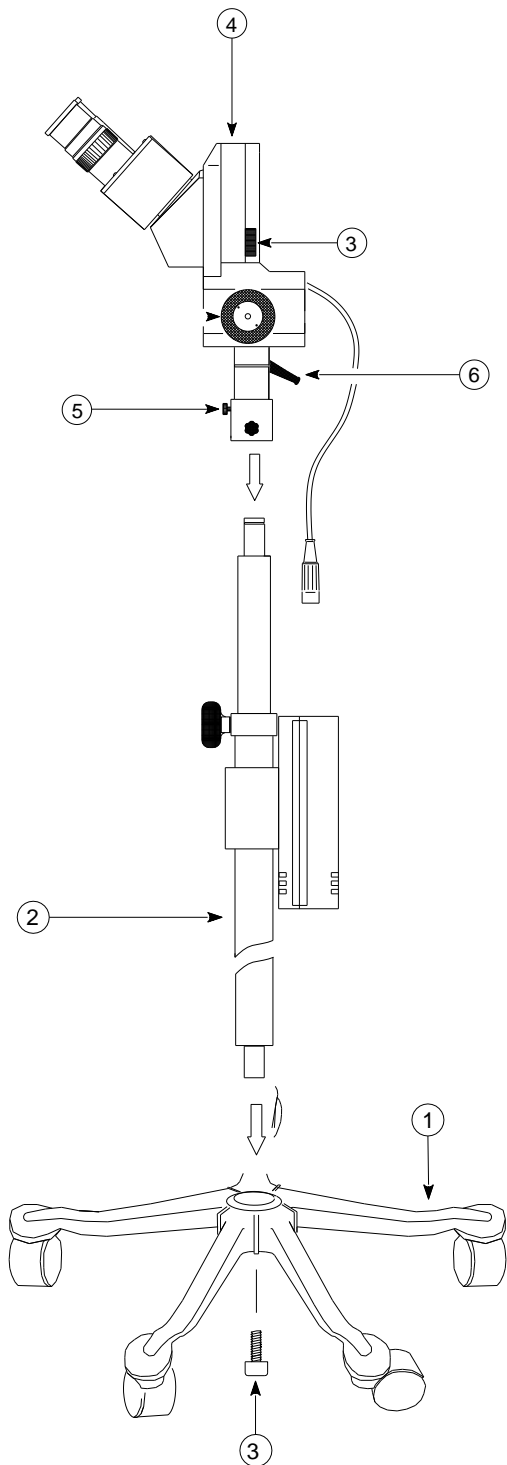
fig. 2

- 4 Insert the stand (2) (fig.2) on the base (1) fixing it with the screw (3) (use the wrench standard supplied)
- 5 Assemble the optical body (4) to the stand screwing in the small grip handle (5)
- 6 Connect the plug (8) in the socket (9).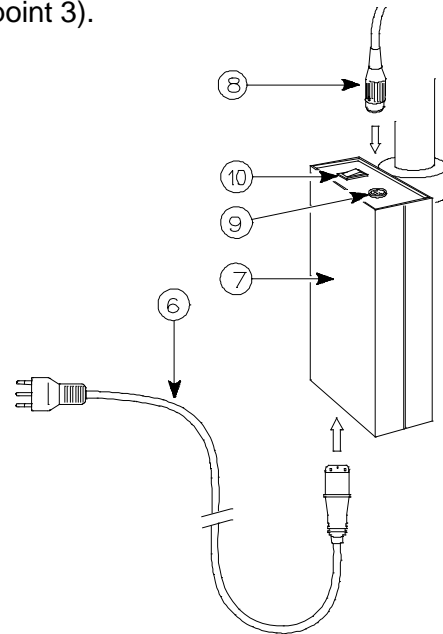

Verify that the mains voltage is the same prescribed in the specifications.

- 7 Connect the power cable (6) to the power unit (7) and therefore to the mains supply.

IV. Using

7. By (8) knob fig.1, set the wished vision height.
8. Regulate the pan-tilt resistance with the (6-7) knobs, fig. 1.
9. Turn "ON" the power unit (10) fig. 2 .
10. Using a plane surface object to observe, set the colposcope at an approximate focus position. Find the best focus image with focusing knob (5) fig.1.
11. Set the correct interpupillary distance removing or approaching the eyepieces with both the hands until the two images overlap in one circle.
12. Diopter compensation: focus on the object looking through the left eyepiece. Looking through the right eyepiece focusig the object turning the diopter compensation ring (2) fig.1.

V. Maintenance – Care

Fuses replacing

Pull out the power cord from the power unit and open the fuses shell as shown in (1) fig.3

Warning: Use only fuses complying with the original. See Specifications.

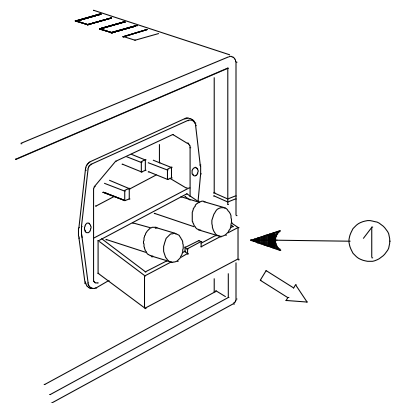
fig.3

Care


- The quality of the image depends mostly on the cleanliness of the eyepieces and of the front protection filter. To remove dirty from these, use a soft paintbrush. For persistent spots it is advise to use a wed of cotton wool soaked in alcool.

The mechanical parts can be cleaned with up with a soft paint-brush. Keep the colposcope far from corrosive substances.

Always keep the colposcope covered with its dust cover when not in use.

The quality of the materials employs in the construction will guarantee the best performances for several years. If Your instrument have not to work correctly, don't hesitate to contact Your CENTREL agent.

VI. Electrical parts

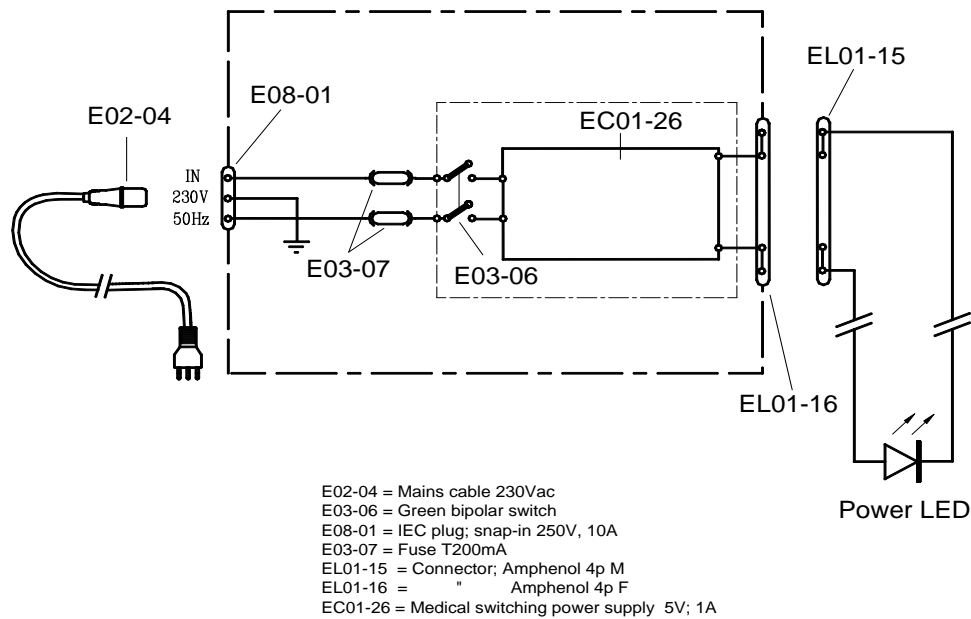VII. Specifications

Eyepieces	10x	15x
Magnification	6,2	9,3
Field of view	35mm	24mm
Working distance	280mm	
Interpupillary distance	48 – 77 mm	
Diopter compensation	±5 on right eyepieces	
Height of vision	73 – 104 cm	
Fine focus	36 mm	
Filter	Green	
Lighting device	Power LED 4000°K	
Fuses (two)	T200mA	
Classification	I type B	
Complies with	IEC-601-1 93/42 CEE	
Main supply	90-240Vac, 50/60Hz	
Weight	5 kg	
Mark		

Specifications could be changed without notice

VIII. Optionals

CODE	DESCRIPTION
313114	MC1 analogue camera to fit directly on one eyepiece.
317014	Monitor 19" Wide, white, HDMI and video PAL input,
513202	Video cable BNC-RCA, 3m.
313103	DL1 USB digital camera 1.3Mpx, to fit on eyepiece
513015	Eyepieces (two) WF 15X.
513102	Adapter to fit a "C" mount camera on eyepiece