GIMA

TAMPONI COTONE
COTTON STICKS
TAMPONS EN COTON
BAUMWOLLTUPFER
TAPONES ALGODÓN
PALITOS COM ALMOFADA DE ALGODÃO
TAMPION BAMBAKIOY
صمامات من القطن

MANUALE D'USO E MANUTENZIONE
USE AND MAINTENANCE BOOK
INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN
BETRIEBS UND WARTUNGS ANWEISUNGEN
MANUAL DE USO Y MANTENIMIENTO
MANUAL DE USO E MANUTENÇÃO
ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ ΚΑΙ ΣΥΝΤΗΡΗΣΗΣ
دليل الإستعمال والرعاية

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COTTON STICKS

Cotton applicator swab indicate for cleaning, drying, wiping, application of solutions.

The product shall be used by qualified medical personnel only.

Do not expose to sources of heat. Use only if the packing is not damaged. Extract the device from the package immediately before use. Before use, visually examine the device to check its integrity.

Intended use: cotton stick is used for cleaning, drying, wiping, application of solutions.

Indications: it is particularly indicated for gynecologists and obstetrics. It can be used to remove the cervical mucos, pad bleeding during day surgery operations, for the disinfection of the cervix, vagina and vulva and during colposcopy for the application of 5% acetic- acid solution or Lugol's solution to the cervix.

Contraindications: none.

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
Directions for use: hold the stem when wrapping; to apply solutions dip the Stick cotton in the solution and wait for a while to help the absorption; apply to the mucosae with moderate pressures.

The device IS NOT STERILE!

The device is SINGLE-USE. Do not clean for a subsequent use!

The product must be disposed according to the local provisions on dangerous medicines disposal.

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