LAMPADA LUXIFLEX - su carrello
LOUPE LUXIFLEX - trolley
LAMPE LUXIFLEX - stand mobile
LÁMPARA LUXIFLEX - con ruedas

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FEATURES

Type of lighting: 30754 halogen 35 Watt - 30756 LED 10 Watt

Lamp life: 5000 h halogen, 25000 h LED

Light intensity at 50 cm (lux) 50,000 halogen, 15,000 LED

Colour temperature: 4,000 K halogen, 3,000 K LED

Class II

PRESCRIPTIONS

The use of this product is reserved to qualified medical personnel only.

Do not use the equipment in case it is damaged. Apply to your retailer.

Avoid precarious repairs. Repairs shall be carried out with original spare parts only, which shall be installed according to the intended use.

Since the product is made of corrosion-proof materials suitable for the environmental conditions foreseen for its normal use, does not require special care, however it is necessary to store it in a closed place making sure that is protected from dust and dirt to assure its hygienic conditions. Moreover, it is recommended to store the product in a place which can be reached easily by the personnel in case of necessity.

UNPACKING

Always remember that packing elements (paper, cellophane, stitches, adhesive tape, etc.) can cut and/or hurt if they are not carefully handled.

They shall be removed with adequate means and shall not be left at the mercy of irresponsible persons; the same is valid for tools used to remove packages (scissors, knives, etc.).

After opening the packages, first of all it is necessary to check all pieces and parts composing the product. Check that they are all present and in perfect conditions.

INSTALLATION

Wheel base with 3 brakes

Very Important: When assembling the legs, remember that for better efficiency it is recommended not to install the three wheels with brakes on adjacent sides.

Insert piece (2) totally into the base (3) and press down. Insert the legs with the wheels (4) through the base slot (3) until it completely fits, and if necessary helping with a nylon hammer.

Fit piece (5) into the interior leg slots (4) by pressing down. Put the screw (7) through the washer (6) and insert the screw into piece (5) to connect the set.

Join the pole (1) directly with the screw (7) without tightening the pole (1) with the piece (2) so as not to damage the paint. Hold the pole (1) firmly and turn the screw (7) with a wrench or a scanner until the whole set is tight.

MAINTENANCE

The product has been conceived to last forever, it is resistant to most part of chemicals except for methyl ethyl ketones (M.E.K.), diluents and nail polish removers.

Do not use abrasive materials to clean the lens as they could suffer from permanent damages.

Do not dip in water! While cleaning, make sure not to wet the light bulbs, the cable and the plug with water or other liquids.

The product can be cold cleaned with a disinfectant and a sponge.

Before removing the lamp, it is necessary to make sure that the instrument is off for some minutes to avoid the risk of burning.

To change the lamp

- Disconnect from electrical.
- Open the lamp cover-ring by pressing it (Fig. 1).
- Change the lamp by pulling it straight out (Fig. 2).
- Fit the new bulb by pressing it straight up into the lamp holder.
- Re-assemble the cover-ring (Fig. 1).

***Disposal:** The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.*

For further information on recycling points contact the local authorities, the local recycling center or the shop where the product was purchased. If the equipment is not disposed of correctly, fines or penalties may be applied in accordance with the national legislation and regulations.

GIMA WARRANTY CONDITIONS

Congratulations for purchasing a GIMA product.

This product meets high qualitative standards both as regards the material and the production. The warranty is valid for 12 months from the date of supply of GIMA.

During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included.

All components subject to wear are not included in the warranty.

The repair or replacement performed during the warranty period shall not extend the warranty.

The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use.

GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electro-magnetic fields, radio interferences, etc.

The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed.

The defected products must be returned only to the dealer the product was purchased from. Products sent to GIMA will be rejected.

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