GIMA

Gima S.p.A. – Via Marconi, 1 – 20060 Gessate (MI) Italy
Italia: tel. 199 400 401 – fax 199 400 403
Export: tel. +39 02 953854209/221/225 fax +39 02 95380056
gima@gimaitaly.com – export@gimaitaly.com

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Thank you for purchasing this product. To prevent damage to your product or injury to yourself or to others, read the following safety precaution in their entirety before using this equipment. Keep these safety instructions where all those who use the product will read them.

PRODUCT CARE

1. Do not expose your product to liquid or humidity.
2. Do not touch the surface of lens with hand or sharp objects.
3. Do not shine the light directly into the patient's eyes.
4. Ensure that the headband power cord is located in the clips provided. The headlight body should not be left covered with a cloth or similar.
5. If it is not used for a long period, please take the batteries out.

* SAFETY MARKS USED:

Type BF

This icon marks warnings, information that should be read before using your product to prevent possible injury.

PURPOSE AND FEATURES OF THE PRODUCT

PURPOSE

30890 headlight is used for examination and surgical operation in multifarious microsurgery.

FEATURES

- Light Source: Our LED lamp without the heat of traditional Halogen headlight. With a color temperature of 6000 Kelvin, this light supplies bright, white, shadow-free light allowing the doctor to see tissue characteristics without distortion.
- Enjoy the flexibility of operating the headlight using the rechargeable Lithium battery, can use without the direct power source. It is one of the most comfortable lightweight headlights available on the market and suitable for all kinds of surgery.
- The headlight could be used when it is charging.
- Has low-battery indicator light.
- Adapts to any Binocular Loupes of GIMA
- Smart design, convenient for mobile diagnosis.

OPERATING INSTRUCTIONS

Attach the power cord by means of the clip provided. Place the instrument on your head and adjust for width and height to that it fits comfortably.

Connect the Headlight to the power source. Switch it on and use the headband right or left brightness control knob to select the required brightness.

Loosen the fixing screw to adjust the light head up and down and backwards or forwards. After adjustment, re-tighten the screw. The illuminated field of view can be selected by means of the slide control. There are two battery indicator lights on battery box, when the yellow lamps light meaning the battery should be charged or another new battery should be used.

If you wish to continue the exam without pausing to recharge the battery, simply plug in the transformer to the main supply.

MAINTENANCE

BATTERY CHARGING AND REPLACING

There are two battery indicator lights on battery box, when the yellow lamps light meaning the battery should be charged or spare battery should be used. If there is only a yellow indicator lighting and flashing, the headlight will switch off automatically to avoid excessively discharged.

If you have the necessity to continue the exam without pausing to recharge the battery, simply plug in the transformer to the main supply. You can use the headlight while the battery is being charged. The battery indicator will be displayed from yellow to green when the battery fully charged.

Caution: Please use the appropriate battery. The battery should never be placed in municipal waste. Please check local regulations for disposal of batteries.

BATTERY USE AND CARE

The capacity of Lithium battery will be changed by it's using condition. For maximum battery capacity, use the battery in room temperature (25 °~40 °). Do not expose the battery to extreme temperatures, if the battery is used in high or low temperatures, the battery capacity will be reduced.

CLEANING AND STERILIZATION

- The outer surface of the instrument may be cleaned with wet cloth, the remaining stains can be cleaned off with the mixture of 50% alcohol and 50% distilled water. Do not wipe with any corrosive detergent lest that the surface should be damaged.
- Cleaning the lenses: Glass-cleaning paper or a drop of liquid solvent (50% alcohol and 50% aether) may be used to clean the lens, wipe it carefully. If there is dust stained on the lens, blow them with a blowball or brush them with a dust pen.

Caution: Be careful to prevent the solvent into the instrument.

TECHNICAL SPECIFICATION

Optical specifications		
Illumination light spot at 420 mm	20-80 mm	
Illumination in 250 mm working distance	≥ 60000 lx	
Adjustable range		
Fluctuation distance	12.5 mm	
Angle range	Vertical ±25°	
Weight		
Headlight (excluding head band)	69 g	
Electrical specifications		
Illumination Source	Extremely bright light, 5 W LED, life 20000 hours continuous burn, do not need to change the LED lamp.	
Battery	7.4 V Lithium battery	
Battery Run-time	Not less than 7 hours continuous run-time on a fully charged battery	
Rating Voltage	100-240 VAC, 50 Hz-60 Hz	
Maximum Input Power	15 VA	
Power Adapter Output	DC: 12V DC, 1250mA	
Electrical Safety Standard	IEC60601-1 EN60601-1-2, class II Type BF equipment	
Environment requirements		
Use Condition	Environment Temperature	+10 °...+40 °
	Relative Humidity	30%...75%
	Atmospheric Pressure	700 hPa...1060 hPa
Shipping and Storage	Environment Temperature	-40 °...+55 °
	Relative Humidity	10%...90%
	Atmospheric Pressure	500 hPa...1060 hPa

 **Class I Medical Device**

Disposal: The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment. For further information on recycling points contact the local authorities, the local recycling center or the shop where the product was purchased. If the equipment is not disposed of correctly, fines or penalties may be applied in accordance with the national legislation and regulations.

GIMA WARRANTY CONDITIONS

Congratulations for purchasing a GIMA product. This product meets high qualitative standards both as regards the material and the production. The warranty is valid for 12 months from the date of supply of GIMA. During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included. All components subject to wear are not included in the warranty.

The repair or replacement performed during the warranty period shall not extend the warranty. The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use. GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electromagnetic fields, radio interferences, etc.

The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed. The defected products must be returned only to the dealer the product was purchased from. Products sent to GIMA will be rejected.

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GIMA S.p.A.