SPECCHI FRONTALI LUX
LUX FOREHEAD MIRRORS
MIROIRS FRONTAUX LUX
STIRNSPIEGEL FRONTALI LUX
ESPEJOS FRONTALES LUX
ESPELHOS FRONTAIS LUX
ΚΑΤΟΠΤΡΟ ΜΕΤΩΠΟΥ LUX
LUX مرايا جبين

MANUALE D'USO E MANUTENZIONE
USE AND MAINTENANCE BOOK
INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN
BETRIEBS UND WARTUNGS ANWEISUNGEN
MANUAL DE USO Y MANTENIMIENTO
MANUAL DE USO E MANUTENÇÃO
ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ ΚΑΙ ΣΥΝΤΗΡΗΣΗΣ
دليل الإستعمال والرعاية

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FEATURES

The ENT forehead mirrors are local lighting devices with luminous forehead mirror particularly useful to examine the oral cavity, the nose and the ear.

These products are equipped with mirrors made of special enameled glass and are assembled on a practical helmet, adjustable and adaptable to the operator's head.

The following models are available:

31108 LUX-55 battery head mirror (2 x 3V CR-123A) - Ø 55 mm - 6V

31109 LUX-100 battery head mirror (2 x 3V CR-123A) - Ø 100 mm - 6V

31110 LUX-55 AC/DC transformer head mirror - Ø 55 mm - 6V

31111 LUX-100 AC/DC transformer head mirror - Ø 100 mm - 6V

The packaging includes a mirror with head band, a 6V bulb and a power cable and plug-in transformer or a battery compartment (depending on the model).

The GIMA 6 plug-in transformer can be used to precisely adjust the light, instead of the model included in the package.

PRESCRIPTIONS

- The mirrors are very delicate and shall be handled carefully.

The product shall be used by qualified personnel only.

Do not use the equipment in case it is damaged. Apply to your retailer. Avoid precarious repairs. Repairs shall be carried out with original spare parts only, which shall be installed according to the intended use.

Broken bulbs or mirrors can cause injuries or cut. Carefully handle and immediately replace.

Since the product is made of corrosion-proof materials suitable for the environmental conditions foreseen for its normal use, does not require special care, however it is necessary to store it in a closed place making sure that is protected from dust and dirt to assure its hygienic conditions. Moreover, it is recommended to store the product in a place which can be reached easily by the personnel in case of necessity.

UNPACKING

Always remember that packing elements (paper, cellophane, stitches, adhesive tape, etc.) can cut and/or hurt if they are not carefully handled.

They shall be removed with adequate means and shall not be left at the mercy of irresponsible persons; the same is valid for tools used to remove packages (scissors, knives, etc.).

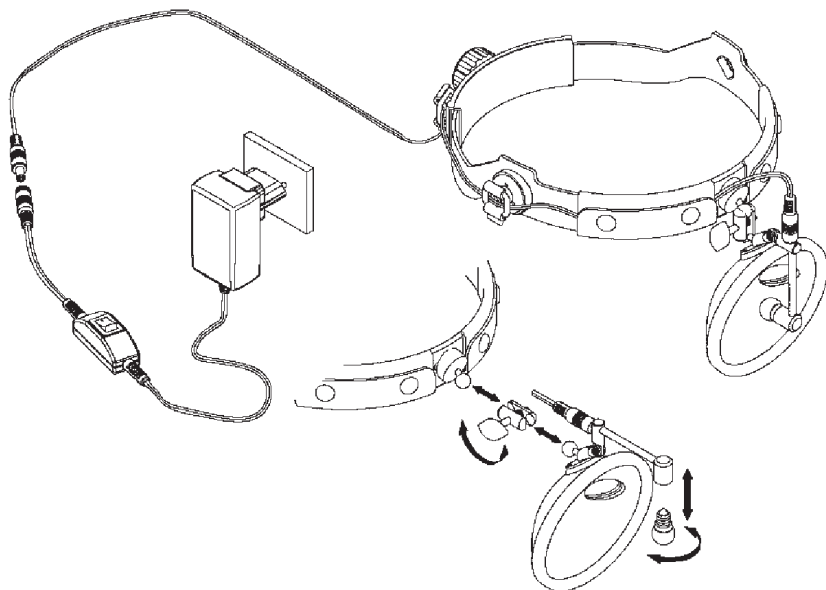
After opening the packages, first of all it is necessary to check all pieces and parts composing the product. Check that they are all present and in perfect conditions.

INSTALLATION


- Before using the product, it is possible to adjust the helmet to the head size. To do this, make use of the opposite adjusting screw located on the back allowing tightening or loosening the headband.

- The devices come with mirrors in two diameters, 100 mm and 55 mm. The device can be powered by either a direct mains adaptor or a battery box.
- The product is available in the version with external adaptor or the version with battery compartment

Installation of model with switching adaptor

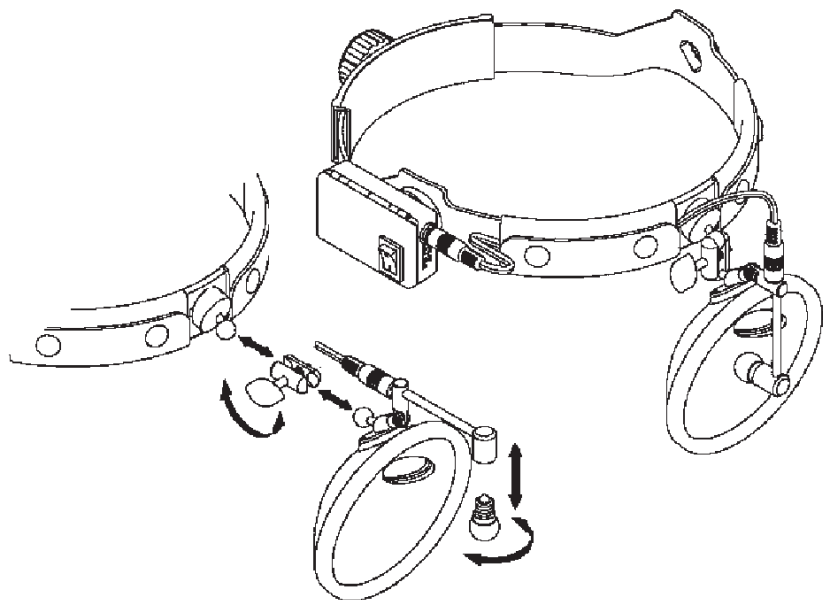Installation of model with battery compartment

- Once the system is setup and light is turned on, the diameter of the spot can be increased or decreased by moving the bulb towards or away from the mirror.

MAINTENANCE

The product has been conceived to last forever; it is resistant to most part of chemicals except for methyl ethyl ketones (M.E.K.), diluents and nail polish removers.

- Handle the product with great care, avoid knocking the mirror, which could break and cause damages. Do not touch the mirror with fingers or objects that could damage or soil it.

Do not use abrasive materials to clean the mirror, which could cause permanent damages.

- In case of broken bulb, unscrew the bulb delicately and replace it with a new one (it is advisable to use only original spare bulbs GIMA).

Before removing the bulb, make sure that the instrument has been turned off for some minutes to avoid the risk of burning.

- Cold clean the helmet with some disinfectant and a sponge.
- Clean the mirror with an adequate detergent and a cotton cloth.

***Disposal:** The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.*

For further information on recycling points contact the local authorities, the local recycling center or the shop where the product was purchased. If the equipment is not disposed of correctly, fines or penalties may be applied in accordance with the national legislation and regulations.

GIMA WARRANTY CONDITIONS

Congratulations for purchasing a GIMA product.

This product meets high qualitative standards both as regards the material and the production.

The warranty is valid for 12 months from the date of supply of GIMA.

During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included.

All components subject to wear are not included in the warranty.

The repair or replacement performed during the warranty period shall not extend the warranty.

The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use.

GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electro-magnetic fields, radio interferences, etc.

The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed.

The defected products must be returned only to the dealer the product was purchased from.

Products sent to GIMA will be rejected.

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