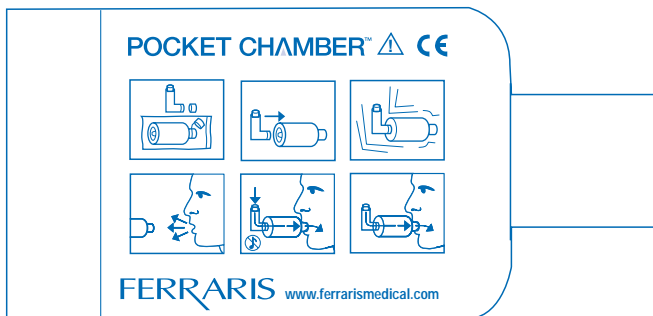
FERRARIS
CardioRespiratory

POCKET *Valved Aerosol Holding Chamber* CHAMBER™

for use with Metered Dose Inhalers
voor gebruik met inhalers met afgepaste dosis
zur Verwendung in Verbindung mit dem Dosierinhalator per aerosol dosati
per aerosol dosati
para utilização com Inaladores de Doses Calibradas
para utilizar con inhaladores dosificadores
pour utilisation avec les inhalateurs doseurs
Όλζύλú Doz Inhaler kullanimi için
για χρήση με δοσιμετρικές συσκευές εισπνοών

INSTRUCTIONS FOR USE
GEBRUIKSAANWIJZING
GEBRAUCHSANLEITUNG
ISTRUZIONI D'USO
INSTRUÇÕES DE UTILIZAÇÃO
INSTRUCCIONES DE USO
MODE D'EMPLOI
KULLANIM AÇIKLAMALARI
ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ

treat-asthma.com

ferrarismedical.com

Contents

English	3
---------	---

Nederlands	4
------------	---

Deutsch	5
---------	---

Italiano	6
----------	---

Português	7
-----------	---

Español	8
---------	---

Français	9
----------	---

Türkçe	10
--------	----

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
For Use with Metered Dose Inhalers


Before using the POCKET CHAMBER, please read the instructions carefully.

The purpose of the POCKET CHAMBER is to reduce the aerosol medication dose from the Metered Dose Inhalers (MDI) by dispersing large aerosol particles to a dose that is more effectively inhaled.

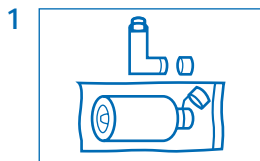
Instructions for Use

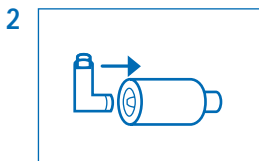
Remove POCKET CHAMBER from bag. Remove caps from both the MDI and the POCKET CHAMBER.

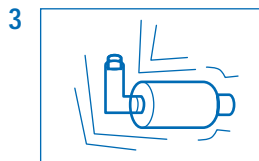
Insert the MDI into the flexible MDI end of the POCKET CHAMBER.


Shake the assembly vigorously.

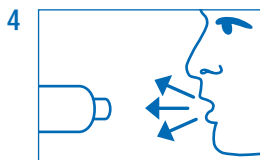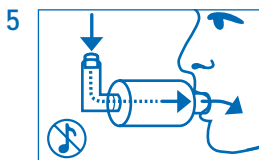Exhale slowly and deeply then place the POCKET CHAMBER in your mouth.

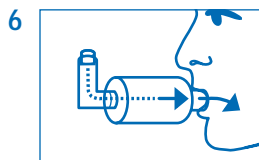Depress the MDI canister and spray one puff into the POCKET CHAMBER while inhaling slowly, so that you do not activate the whistle.

Continue to breathe in slowly and fully.

7 Try to hold your breath for 5 to 10 seconds.

8 Exhale slowly.

9 Repeat Steps 3 - 8 as many times as instructed by your physician.

Technical Information

The POCKET CHAMBER helps to deliver the MDI medication. The POCKET CHAMBER's recessed baffle plate reduces the non-respirable (non-therapeutic) particle count that would otherwise deposit in your throat or mouth. At the same time, the POCKET CHAMBER delivers an adequate dose of respirable (therapeutic dose) MDI aerosol medication that will be easily inhaled into the lungs. This process helps to reduce unwanted side effects and promotes an effective treatment. If you should have any questions or problems using the POCKET CHAMBER, please contact your physician.

Warnings and Cautions

The POCKET CHAMBER should be replaced if:

1. The body is cracked
2. The flexible MDI end has split
3. The POCKET CHAMBER's recessed baffle plate is cracked or missing

Cleaning

With repeated use, medication powder will build up on the POCKET CHAMBER's inner surfaces. This may interfere with the performance of the device. The device along with the MDI plastic elbow should be washed at least once a week using warm water and a liquid dishwashing detergent. Place the whole unit in warm soapy water before trying to remove flexible MDI end. Then wash the unit, rinse with warm clear water and allow to air dry.

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FERRARIS MEDICAL Ltd