# GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italy  
Italia: tel. 199 400 401 - fax 199 400 403  
Export: tel. +39 02 953854209/221/225 fax +39 02 95380056  
gima@gimaitaly.com - export@gimaitaly.com  
[www.gimaitaly.com](http://www.gimaitaly.com)

**Barella GIMA 4**  
**GIMA 4 Stretcher**  
**Brancard GIMA 4**  
**Krankentrage GIMA 4**  
**Camilla GIMA 4**  
**Maca GIMA 4**  
**Φορέο GIMA 4**  
**نقالة مرضى GIMA 4**

MANUALE D'USO E MANUTENZIONE  
USE AND MAINTENANCE BOOK  
INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN  
BETRIEBS UND WARTUNGS ANWEISUNGEN  
MANUAL DE USO Y MANTENIMIENTO  
MANUAL DE USO E MANUTENÇÃO  
ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ ΚΑΙ ΣΥΝΤΗΡΗΣΗΣ

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## FEATURES

The stretcher was designed to make the handling of patients easier indoors or outdoors on regular surfaces with modest inclinations, for extreme or urgent necessities. This stretcher is not designed to accommodate patients and cannot be used as bed. The stretcher must be held by more than one operator (preferably 4) strong enough to carry the overall weight. For the safety of the patient it is necessary to secure the patient to the stretcher using straps, not supplied. The maximum capacity is 130 kg, calculated with the stretcher on a flat surface. The product or any of its parts cannot be used for purposes other than the ones indicated in this manual.

## WARNINGS

*The tubular pieces **B**, the handles **M** and all the closeable parts may generate risk of crushing. Pay attention that hands, feet and any other object is not crushed by these parts. Non-compliance with these warnings and the following suggestions could cause injuries to people. Do not use the stretcher with greasy, dirty or sweaty hands.*

*Do not use the product if damaged. Contact supplier.  
Avoid any temporary repair.*

*Pay attention to any burr on tubular pieces, feet and blocking devices. If visible remove with appropriate tools because they could injury people during use.*

*Do not crash against walls, steps or banisters, because the stretcher may fold or tilt over unexpectedly. Do not store the stretcher leaning it only on the central part, make sure the feet are completely set on a flat surface.*

Since the product is made of corrosion-proof materials suitable for the environmental conditions foreseen for its normal use, does not require special care, however it is necessary to store it in a closed place making sure that is protected from dust and dirt to assure its hygienic conditions. Moreover, it is recommended to store the product in a place which can be reached easily by the personnel in case of necessity.

## UNPACKING

*Always remember that packing elements (paper, cellophane, stitches, adhesive tape, etc.) can cut and/or hurt if they are not carefully handled.*

*They shall be removed with adequate means and shall not be left at the mercy of irresponsible persons; the same is valid for tools used to remove packages (scissors, knives, etc.).*

After opening the packages, first of all it is necessary to check all pieces and parts composing the product. Check that they are all present and in perfect conditions.

## OPERATION

*Pay attention to the hinged or mobile elements. Risk of crushing.*

*Also pay attention to the use of the handles **M** they must be engaged when the stretcher is fully open.*

Always secure the patient to the stretcher so that he/she does not fall due to unexpected crashes or steep slopes. If the stretcher has to move along steep slopes (ex. stairs) try to counter-balance the slope by keeping the stretcher as horizontal as possible. Do not leave the stretcher unattended with patient secured to it.

**MAINTENANCE**

Clean the stretcher without using abrasive products or solvents (we suggest using soap and water).

After washing the stretcher dry every part thoroughly.

To disinfect, use a combination of cleaning/disinfecting products. Dilute the disinfectant following the manufacturer's instructions.

Periodically verify the state of wear of the stretcher material **T**, of the handles **B** and hinges **C**.

**GIMA WARRANTY CONDITIONS**

Congratulations for purchasing a GIMA product.

This product meets high qualitative standards both as regards the material and the production.

The warranty is valid for 12 months from the date of supply of GIMA.

During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included.

All components subject to wear are not included in the warranty.

The repair or replacement performed during the warranty period shall not extend the warranty.

The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use.

GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electro-magnetic fields, radio interferences, etc.

The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed.

The defected products must be returned only to the dealer the product was purchased from.

Products sent to GIMA will be rejected.

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