

Mascherina facciale in silicone

REF	DESCRIZIONE
34220	Mascherina N° 0 - neonato/small
34221	Mascherina N° 1 - neonato/large
34222	Mascherina N° 2 - bambino/small
34223	Mascherina N° 3 - bambino/large
34224	Mascherina N° 4 - adulto/small
34225	Mascherina N° 5 - adulto/large

ATTENZIONE

1. La legge federale degli USA impone la vendita del presente dispositivo esclusivamente sotto prescrizione medica.
2. Il paziente deve essere costantemente monitorato durante l'utilizzo del presente dispositivo.
3. Conservare in ambiente pulito e asciutto, lontano dalle fonti di calore e dalla luce.

ISTRUZIONI D'USO

1. Prima dell'uso, controllare che il flusso d'aria passi in modo regolare.
2. Collegare al circuito respiratorio o al dispositivo di rianimazione tramite gli adattatori appropriati.
3. Assicurarsi che tutti i collegamenti siano quelli adeguati e che siano ben saldi.
4. Tenere fermo il dispositivo con la mano o assicurarlo tramite adeguato supporto a fascia per la testa.

PULIZIA E STERILIZZAZIONE

Sterilizzare i componenti usando uno dei seguenti metodi:

- a. Autoclave (max 134°C/273°F)
- b. Ossido di etilene
- c. Soluzione disinfettante comune usata per attrezzature che vanno direttamente a contatto con il paziente.
Lavare abbondantemente con acqua dopo la sterilizzazione.

 **GIMA S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italia - www.gimaitaly.com**
MADE IN P.R.C.

 **0476**

Silicone Face Mask

REF	DESCRIPTION
34220	Face mask N° 0 - infant/small
34221	Face mask N° 1 - infant/large
34222	Face mask N° 2 - child/small
34223	Face mask N° 3 - child/large
34224	Face mask N° 4 - adult/small
34225	Face mask N° 5 - adult/large

CAUTION

1. Federal Law (USA) restricts this device to sale by or on the order of a physician.
2. Patient should be monitored constantly whenever this device is in use.
3. Store in clean dry conditions away from heat and light.

DIRECTIONS FOR USE

1. Check for clear airflow passage prior to use.
2. Connect to breathing circuit or resuscitation device with appropriate fitting.
3. Make sure all connections are compatible and fit firmly.
4. Hand-hold or secure with head strap.

CLEANING AND STERILIZATION

Sterilize the components using one of the following methods:

- a. Autoclave (max 134°C/273°F)
 - b. Ethylene oxide
 - c. Most common disinfecting solutions which are used for equipment coming into contact with the patient.
- Rinse thoroughly with water after sterilization.



GIMA S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italia - www.gimaitaly.com
MADE IN P.R.C.

