# GIMA

**COLLARI**  
**COLLARS**  
**COLLIERS**  
**COLLARES**  
**COLARES**  
*Περιλάμια*

**MANUALE D'USO E MANUTENZIONE**  
**USE AND MAINTENANCE BOOK**  
**INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN**  
**MANUAL DE USO Y MANTENIMIENTO**  
**MANUAL DE USO E MANUTENÇÃO**  
**ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ ΚΑΙ ΣΥΝΤΗΡΗΣΗΣ**

**REF 34600 - 34601 - 34602 - 34610 - 34612 - 34614**

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## FEATURES

First-aid collars GIMA are conceived to guarantee the utmost efficiency, comfort and safety as far as support and correction in first-aid conditions concerns. They are also equipped with an anatomical chin rest and a Velcro fastener, which makes the patient be more comfortable with the correct regulation. The product or its components cannot be used for purposes different from the one specified in the present manual.

## PRESCRIPTIONS

The collar GIMA can be used only upon medical prescription.

Before starting the therapy, it is advisable to identify the most suitable collar to meet each individual requirement and in case of necessity to customise it by means of adjustments in compliance with the doctor's instructions.

*In presence of plagues and infectious states, apply to a doctor for a suitable treatment and ask for indications on the product cleaning!*

*Do not use the product if it is damaged. Apply to your retailer.*

Since the product is made of corrosion-proof materials suitable for the environmental conditions foreseen for its normal use, does not require special care, however it is necessary to store it in a closed place making sure that is protected from dust and dirt to assure its hygienic conditions. Moreover, it is recommended to store the product in a place which can be reached easily by the personnel in case of necessity.

### USE

The device must be correctly positioned on both sides and the patient must feel completely comfortable; furthermore it must not be subject to modifications of its external appearance as long as it is used. The device must be installed by the patient itself, no assistance is needed. In case the patient is not able to position the product by himself or herself, it is necessary to train the person assisting him or her on the correct and safe use of the collar.

In case the patient shows signs of hyperpressure or discomfort due to the device stiffness, immediately remove it until the situation has been completely normalised.

In case the subject himself or herself is not able to show such discomfort or undesirable effects deriving from the presence of the collar, carry out periodical inspections to check the situation.

In case of allergic reactions due to the contact of the collar with the skin (reddening, swelling, oedema, skin irritation, itching) or bacterial infections due to lack of hygiene, apply to the family doctor.

In case the patient's weight changes, it is necessary to change the collar size, apply to your doctor.

The doctor must inform the patient about the device usage times based on the type of pathology. The success of the application depends upon the patient's care to follow the doctor's instructions and the continuous contacts with him.

### MAINTENANCE

The product can be washed with cold or warm running water and neutral soap. Do not use aggressive chemical products (bleach, ammonia, solvents...), which could alter the functionality of the product. Do not expose the product to excessive mechanical stresses to avoid altering its functionality.

Carry out periodical controls to make sure that the device has not suffered from modifications, which could jeopardize its functionality.

The product life can vary according to the therapeutic application; to guarantee a longer life of the product it is recommended to correctly use, maintain and periodically check it.

### GUARANTEE

Thank you for having acquired our product.

This product meets the most stringent requirements regarding the selection of quality manufacturing materials and also the final control. The product has a 12-month warranty, valid from the date it is delivered by GIMA. During the guarantee period free repair and/or replacement of any defective parts due to faulty manufacture will be given, labour, postal, transport and packaging charges etc. are not included. The guarantee therefore excludes components subject to wear and tear such as parts in rubber or PVC, etc. No compensation will be given for loss of use of the product.

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
Furthermore replacement or repairs effected during the guarantee period do not lengthen the duration of the guarantee.

*This guarantee is void in the event of: repairs effected by unauthorised persons or with spare parts not approved by GIMA, damage or defects caused by negligence, blows, abnormal use of the appliance or faulty installation.*

The guarantee will be invalidated if the registration number has been removed, cancelled or altered.

Faulty appliances must be returned only to the retailer where the item was bought. Any item dispatched directly to us will be refused.

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GIMA S.p.A.