**LOGIKODIGIT**

**DM460**

**DM470**

**DM450**

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Venduto a \_\_\_\_\_

Via \_\_\_\_\_ Località \_\_\_\_\_

Hangzhou Sejoy Doctor Electronics & Instruments Co., LTD Building 2, No.202 Zhenzhong Road -  
West Lake Economy & Technology Zone 310030 Hangzhou China

SHANGHAI INTERNATIONAL TRADING CORP. GMBH Eiffestrasse 80, 20537 Hamburg, Germany

Distribuito da / Distributed by:

MORETTI S.P.A.

Via Bruxelles, 3 - Melegnano 20022 Cavriglia (Arezzo) Tel. +39 055 96 2111

www.morettispa.com email: info@morettispa.com

MADE in P.R.C.

## INDEX

CODE.....	pag.19
INTRODUCTION .....	pag.19
INTENDED USE.....	pag.19
REGULATIONS AND DIRECTIVE OF REFERENCE .....	pag.19
GENERAL WARNING .....	pag.19
SYMBOLOLOGY .....	pag.20
GENERAL DESCRIPTION .....	pag.20
- Parts view (DM450) .....	pag.21
- Parts view (DM460) .....	pag.22
- Parts view (DM470) .....	pag.22
- Graphic symbols on display (DM450-460-470) .....	pag.23
OPERATING THE UNIT AND CONNECTING PARTS.....	pag.24
- Installing batteries.....	pag.24
- Using the AC/DC adapter (optional) .....	pag.24
- Applying the armcuff .....	pag.24
BEFORE USE .....	pag.25
CAUTIONS BEFORE USE.....	pag.25
HOW TO USE .....	pag.25
- Quick start test.....	pag.25
- Time/Date Setting.....	pag.26
- Memory group icon settings (DM460-470).....	pag.26
- Reading Memories .....	pag.26
- Last 3 tests average (DM460-470).....	pag.26
- Memory deletion .....	pag.27
- Pressure measurement.....	pag.27
MAINTENANCE.....	pag.29
CLEANING AND DISINFECTION.....	pag.29
CONDITIONS OF DISPOSAL .....	pag.29
Treatment of batteries.....	pag.29
Disposal instructions of waste batteries.....	pag.30
SPARE PARTS AND ACCESSORIES .....	pag.30
TROUBLESHOOTING .....	pag.30
SPECIFICATIONS .....	pag.31
WARRANTY .....	pag.32

**Class I medical and measuring device Legislative decree 24/02/97 n.46  
in accordance with EEC directive 93/42 and its amendments**

**CODES**

- DM450** Digital sphygmomanometer desk type, LCD display 3"  
**DM460** Digital sphygmomanometer desk type, LCD display 4"  
**DM470** Digital sphygmomanometer desk type, LCD display 4,8"

**INTRODUCTION**

Thank you for purchasing LOGIKO sphygmomanometer by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advise you properly

**WARNING!**

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

**INTENDED USE**

The digital sphygmomanometer desk type are measurement devices for systolic and diastolic blood pressure

**REGULATIONS AND DIRECTIVE OF REFERENCE**

The LOGIKO sphygmomanometers by Moretti S.p.A are in accordance to the following regulations:

- EN 1060-1**            Non-invasive blood pressure measuring equipment General requirements  
**EN 1060-3**            Non-invasive blood pressure measuring equipment  
 Supplementary requirements for electromechanical blood pressure measuring systems

**GENERAL WARNINGS**

- Do not use this product or any available optional equipment without first completely reading and understanding this instruction manual.
- Advise with physician or therapist to determinate the correct adjustment and the correct use of the sphygmomanometer
- Keep the packed sphygmomanometer away from heat sources
- Service life - the wheelchair use limit is defined by the wear of the parts
- Do not allow children to play on or operate with the sphygmomanometer

**SYMBOLOLOGY**

 serial number	 warning	 ECREP	 type B	 instruction
 waste disposal	 CEO 0197	 manufacturer		

**GENERAL DESCRIPTION**

**What is Blood Pressure?**

Blood pressure is the pressure exerted on the artery tube while blood flows through the arteries. The pressure measured when the heart contracts and sends blood out of the heart is systolic (highest) blood pressure. The pressure measured when the heart dilates with blood flowing back into the heart is called diastolic (lowest) blood pressure. Blood pressure is measured in mmHg. It is submitted to many fluctuations and it depends on many elements. It is usually lower in summer and higher in winter. Blood pressure can change according to atmospheric pressure and it is affected by physical efforts, temperament, stress, meals, medicines, alcohol, smoking, age..etc...


It is recommended to note down your daily measurements and to consult with your physician who will tell you your normal blood pressure range.

For the above reasons do not worry if you encounter some measurements at high level, consult with your physician who will tell you the point at which you will be considered at risk.


Why measure your blood pressure?

Among the various health problems afflicting modern people, those associated with blood pressure are by far the most common. High blood pressure's dangerously strong correlation with cardiovascular diseases and high morbidity has made measuring blood pressure as a necessity in indentifying those at risk.

**NOTE** Blood pressure can change with the age, it is necessary to consult with a physician to know your normal blood pressure range. Never change drug dosage without consulting with the physician!

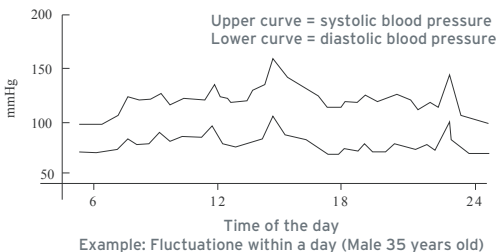
The following scheme has been developed by The World Health Organization (WHO), and it identifies the low- and high-risk blood pressure areas. This standard, however, is a general guideline as individuals' blood pressure varies among different people and different age groups, it is always subjective.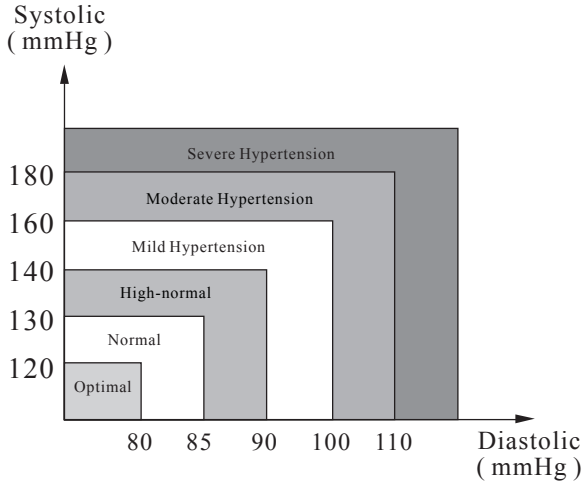

**Parts view (DM450)**


Contents

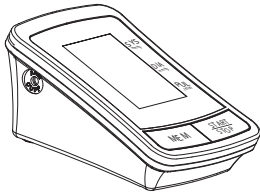
1. Monitor unit

2. Owner's Manual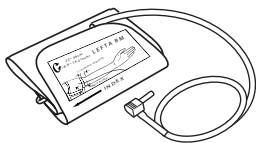


3. Arm Cuff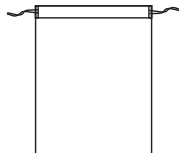

4. Storage Case

**Parts view (DM460)**


Contents

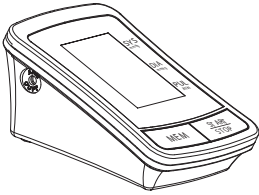
1. Monitor unit

2. Owner's Manual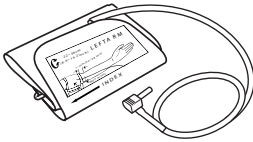


3. Arm Cuff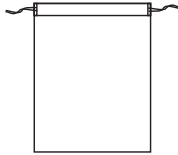

4. Storage Case

**Parts view (DM470)**


Contents

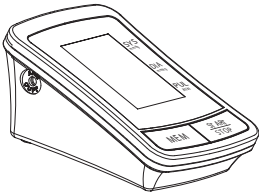
1. Monitor unit

2. Owner's Manual

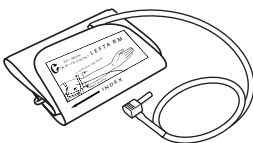
3. Arm Cuff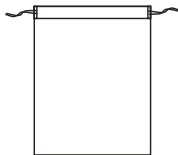



4. Storage Case

Graphic symbols on display (DM450-460-470)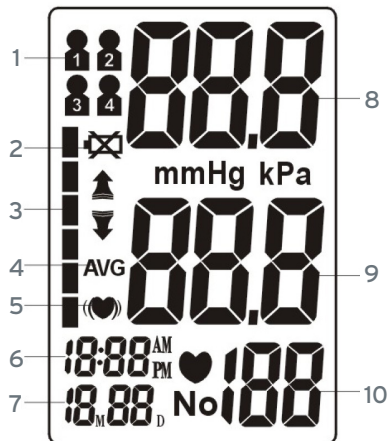

1. Groups of memories (1/2) only mod. DM460-470
2. Low battery indicator
3. WHO blood pressure classification indicator
4. Last 3 measurements average
5. Irregular heartbeat indicator
6. Time
7. Date
8. Systolic blood pressure
9. Diastolic blood pressure
10. Pulse rate/min

**NOTE** The arrow indicator will show your current measurement value according to WHO classification. See the scale here below: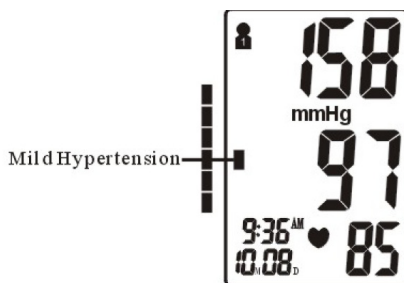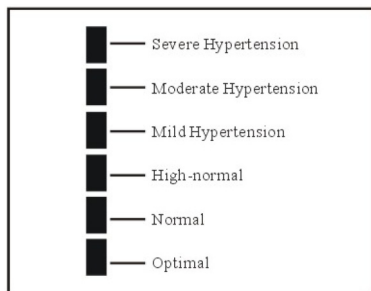


█: Blood Pressure Classification Indicator

**OPERATING THE UNIT AND CONNECTING PARTS:****Installing batteries:**

The battery compartment is situated in the back of the device. Press down slightly and let slide the cover in the arrow direction.


Install 4 AA alkaline 1,5V batteries according to the indications inside the compartment.

Replace the cover of the battery compartment.

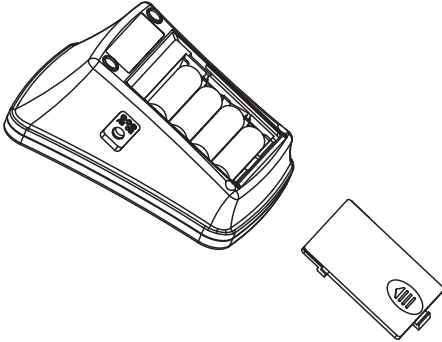
**Using the AC/DC adapter (optional)**

The battery compartment is situated in the back of the device. Press down slightly and let slide the cover in the arrow direction.

Install 4 AA alkaline 1,5V batteries according to the indications inside the compartment.

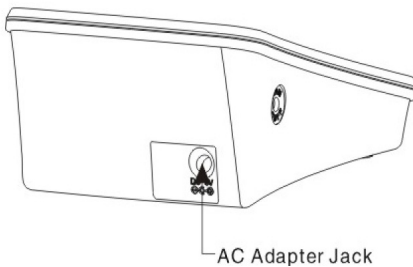**Applying the arm cuff**

Insert the air tube plug into the opening located on the left side of the unit.

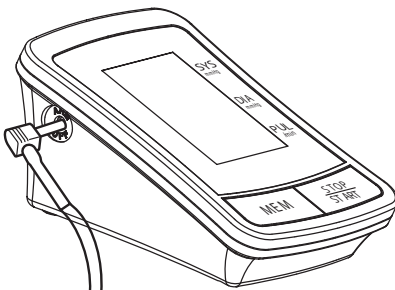## BEFORE USE

- Check if the device is damaged in order to guarantee a safe use for people and things
- Check the correct assembly, paying particular attention to correctly placing batteries and armcuff
- Verify the batteries correctly works, in case substitute them

## CAUTIONS BEFORE USE

- Avoid any type of physical exercise for at least 30 min before the measurement
- Sit in a calm environment for at least 5 minutes before the measurement
- Avoid speaking during the measurement
- Do not move during the measurement
- Do not stand during the measurement. Sit in a relaxed position, rest your elbow and be sure the cuff is at heart level
- During the measurement, avoid electromagnetic interference with other electronic devices
- Wait 3 min before a new measurement
- If you need to measure your blood pressure every day, do it at the same time of the day
- Results comparisons should be made only when the monitor is used on the same arm, in the o same position and at the same time of the day
- This blood pressure monitor is not recommended for people with strong arrhythmia.
- Do not use the device when damage and/or malfunction has occurred
- Remove batteries when the unit is not in use for an extended period of time

## HOW TO USE

**NOTE** install batteries to use digital sphygmomanometer (see “BEFORE USE” section)

### Quick start test

Press START/STOP button and keep pressed until you hear “beep”. The LCD display turns on for a second for a self-test.

At the end the display will show 0 for both, SYS and DIA pressure and this means the device is ready for use.

**NOTE** If there’s still some pressure in the cuff, the indicator in low will flash. Deflate completely the cuff.

### Time/Date settings

#### DM450

- When power off, press “START/STOP” button for 3 seconds to set the Time/Date mode.
- “M” (stands for month) is flashing
- Press “MEM” button for set up
- Press “START/STOP” button to confirm it
- “D” (stands for day) is flashing
- Press “MEM” button for set up
- Set the day (“D”), hour (in AM/PM format)and minute in the same way.
- Press “START/STOP” button for 3 seconds to turn the unit off. All information will be saved

**DM460**

- When power off, press “SET” button to start settings
- The Memory Group icon flashes in the upper left corner (see Memory Group settings)
- Press “SET” button to set the “Time/Date” mode
- “M” (stands for month) is flashing
- Set the month by pressing “M” button
- Press SET again to confirm current month
- “D” (stands for day) is flashing
- Set the day (“D”), hour (in AM/PM format) and minute in the same way
- Press “START/STOP” button for 3 seconds to turn the unit off. All information will be saved

**DM470**

- When power off, press “SET” button to start settings
- The Memory Group icon flashes (see Memory Group settings)
- Press “SET” button to set the “Time/Date” mode
- When the “Yr” icon flashes select M button to set current year
- Press “SET” button again to confirm it
- Set the month, day, hour and minutes in the same way
- Press “SET” button again to set the time format setting mode. Set the time format by adjusting the “M” mode. EU means European time while, US means U.S. time

**Memory group icon settings (DM460-470):**

- When power off, press “SET” button to start settings
- The Memory Group icon flashes
- Press “M” button to choose a group settings (group 1 or 2)
- Press “START/STOP” button to turn the unit off. All information will be saved

**Reading Memories**

- When power off, press “M-MEM” button
- Latest test result group will be shown (see Memory Group settings)
- Press “M-MEM” button to show different test results (up to 60 memories per group)

**Last 3 tests average (DM460-470):**

- It allows checking the average test result from the last 3 readings of the last group used (AVG symbol will appear on display)
- When power off, press “M” button to activate screen display

**Memory deletion**

- DM450**
- Enter the memory reading mode (please see paragraph “Reading Memories”).
  - When memories are displayed, push and hold for 3 seconds START/STOP
  - Memories are erased and the display shows “- - -”
  - Push START/STOP to switch the device off

- DM460-470**
- Enter the memory reading mode (please see paragraph “Reading Memories”).
  - When memories are displayed, push and hold for 3 seconds SET
  - Memories are erased and the display shows “- - -”
  - Push START/STOP to switch the device off

**NOTE** this procedure must be independently applied on both memories

**Pressure measurement:**

1. The place and surroundings should be warm and quite
2. Sit in front of a table or desk, where you can easily rest your arm


3. Wait some minutes before start measuring, in order to make pulse and blood flow stabilize
4. Remove any garment from left arm, where cuff will be worn

**NOTE** Be sure that any piece of clothing do not tighten the arm where pressure measurement is taken

5. Wrap the cuff around the naked arm, so that the central part of the cuff matches the brachial artery. The lower part of the cuff should be set 1-2 cm above the elbow joint. Tighten the cuff in order for it to be adherent to the arm. It must be possible to insert a finger between cuff and arm

**NOTE** do not tighten too much, otherwise blood flow will excessively reduce.

6. Lay the elbow on the table, so that keeps the same level of heart, with the palm of hand facing up

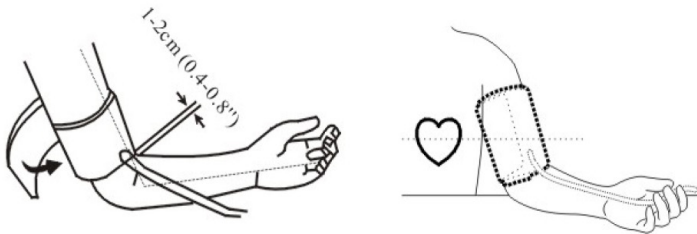7. Push START/STOP button to start measurement (the device will switch on; after an initial test making the display show all the characters, the cuff will automatically inflate)
 

**NOTE** the cuff's initial inflation pressure is 190 mmHg. If the patient's Systolic pressure is higher than the cuff's initial inflation pressure, the device will automatically further inflate the cuff
8. When the cuff is inflated, the internal compressor will stop and the pressure measurement will start, slowly deflating the cuff (during deflating, the symbol ♥ will blink on the display to show heart rate)
9. Stand totally still during the whole pressure measurement
10. When the measurement is finished, the following results will be shown on the display:
  - Systolic pressure (SYS)
  - Diastolic pressure (DIA)
  - Beats per minute (PULSE)
  - Classification of values according to standard WHO/OMS (World Health Organization)
  - Indicator of irregular beats (Arrhythmia) (♥)

**NOTE** this symbol will be shown only if the device detects two or more irregular beats during measurement.

### MAINTENANCE

The LOGIKO devices by Moretti S.p.A are checked carefully and supplied with EC mark, once launched on the market.


For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every year.

In case of repair, use only original spare parts and accessories.

### CLEANING AND DISINFECTION

- **DO NOT immerse the device!**
- **Cleaning:** use only a damp cloth. Pay attention to the display
- **Disinfection:** use a cleanser, avoiding the liquid seep into the device

**NOTE** Never use acids, alkalis or solvents such as acetone

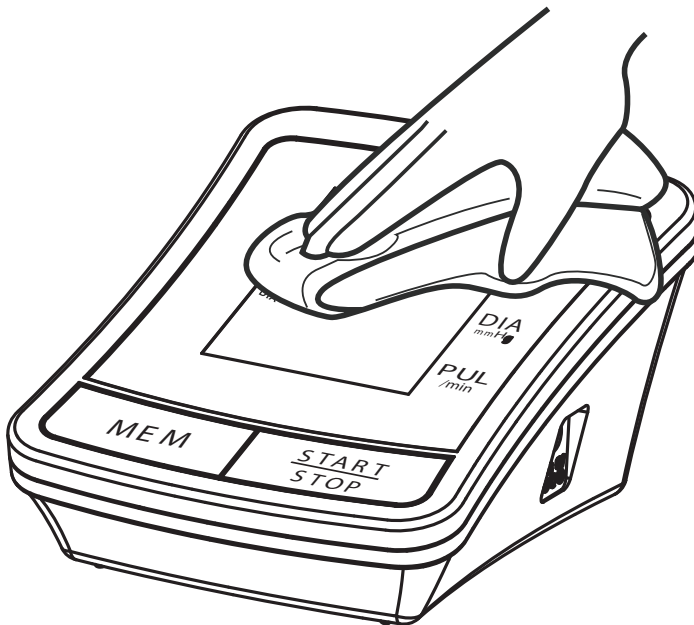**CONDITIONS OF DISPOSAL**

**-General conditions of disposal**

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the sphygmomanometer in the appropriate disposal areas for recycling.

**TREATMENT OF BATTERIES (Directive 2002/96/EC)**

At the end of its life, the product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment or at retailers that provide this service. By ensuring these batteries are disposed of correctly, you will help prevent potentially negative consequences for the environment and human health which could otherwise be caused by inappropriate waste handling of the battery. To remark the need to dispose of electrical equipment separately, the products is marked with crossed mobile waste bin

**DISPOSAL OF WASTE BATTERIES - Directive 2006/66/EC)**

This symbol on the battery or on the packaging indicates that the battery provided with this product shall not be treated as household waste. By ensuring these batteries are disposed of correctly, you will help prevent potentially negative consequences for the environment and human health which could otherwise be caused by inappropriate waste handling of the battery. The recycling of the materials will help to conserve natural resources. At the end of their life hand the batteries over to the applicable collection points for the recycling of waste batteries. For more detailed information about recycling of this product or battery, please contact your local Civic Office, your household waste disposal service or the shop where you purchased the product.

**SPARE PARTS AND ACCESSORIES:**

- DR205- power supply
- DR210- adult cuff
- DR215-obese cuff

**TROUBLESHOOTING**

PROBLEM	POSSIBLE CAUSE	SOLUTION
<b>Blood pressure results are not reliable</b>	<ol style="list-style-type: none"> <li>1. Cuff is too tight or not properly positioned on the arm</li> <li>2. Body or device movements during measurement</li> </ol>	<ol style="list-style-type: none"> <li>1. Firmly reposition cuff as illustrated in point 5, paragraph Pressure Measurement</li> <li>2. Stand still and relaxed during measurement, avoiding speaking</li> </ol>
<b>"ERR" displayed</b>	<ol style="list-style-type: none"> <li>1. Cuff fails to inflate properly</li> <li>2. Improper operation</li> <li>3. Pressurization is over 300mmHg</li> </ol>	<ol style="list-style-type: none"> <li>1. Check connections between cuff and device</li> <li>2. Read user manual carefully and do the operation properly</li> <li>3. Read user manual carefully and do the operation properly</li> </ol>

**SPECIFICATIONS**

<b>PRODUCT DESCRIPTION</b>	Arm type fully Automatic Blood Pressure monitor
<b>MODEL</b>	DM450 - DM460 - DM470
<b>LCD DIGITAL DISPLAY</b>	DM450 62.7 mm x 46.4 mm (3") DM460 84.1 mm x 55.1 mm (4") DM470 (back light) 102.1 mm x 68.9 mm (4.8")
<b>MEASUREMENT METHOD</b>	Oscillometric method
<b>MEASUREMENT RANGE</b>	Pressure 0 mmHg / 300 mmHg Pulse 30 to 180 Beats/Minute
<b>MEASUREMENT ACCURACY</b>	PRESSURE 3 mmHg PULSE 5%
<b>PRESSURIZATION</b>	AUTOMATIC PRESSURIZATION
<b>MEMORY</b>	DM450 - 120 memories DM460 - 120 memories in two groups with date and time DM470 - 120 memories in two groups with date and time
<b>FUNCTION</b>	Irregular beat detection WHO/OMS classification indicator Last 3 result average - AVG (only for DM460-470) Automatic Power off
<b>POWER SOURCE</b>	4 batterie alkaline tipo AA da 1,5V Alimentatore Ac-Dc 6.0Vdc 1000mA ( OPTIONAL)
<b>BATTERY LIFE</b>	APPROXIMATELY 2 MONTHS AT 3 TESTS PER DAY
<b>UNIT WEIGHT</b>	DM450 - approx. 390 g (excluding batteries) DM460 - approx. 405 g (excluding batteries) DM470 - approx. 410 g (excluding batteries)
<b>UNIT DIMENSIONS</b>	DM450 - 110 x 99 x 66 mm (L x W x H) DM460 - 155 x 110 x 70 mm (L x W x H) DM470 - 155 x 110 x 70 mm (L x W x H)
<b>CUFF CIRCUMFERENCE</b>	22.0 cm - 36.0 cm
<b>OPERATING ENVIRONMENT</b>	Temperature 5°C - 40°C Humidity < 85%RH Pressure Atmospheric Pressure
<b>STORAGE ENVIRONMENT</b>	Temperature 10°C - 55°C Humidity < 95% RH Pressure 525mmHg - 795mmHg
<b>CLASSIFICATION</b>	-Internal Powered Equipment. Type B  -Cuff is the Applied Part

### WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to.

The correct intended use is specified in this manual.

Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

### RECHARGEABLE BATTERIES WARRANTY

The original batteries and the spare batteries are covered by a 90 days warranty during a period of 6 month, with regards to manufacturing defeat or to what concern the law regulation. If charge batteries are unused for more than 3 consecutive months, the warranty is void. If exhausted batteries are unused for more than 3 consecutive days, the warranty is void.

### REPAIRING

#### WARRANTY REPAIR

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

#### REPAIR NOT COVERED BY WARRANTY

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

### SPARE PARTS

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

### EXEMPT CLAUSES

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, , including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

## WARRANTY CERTIFICATE

**Product** \_\_\_\_\_

**Purchasing date** \_\_\_\_\_

**Authorized dealer** \_\_\_\_\_

**Street** \_\_\_\_\_ **Place** \_\_\_\_\_

**Sold to** \_\_\_\_\_

**Street** \_\_\_\_\_ **Place** \_\_\_\_\_

Hangzhou Sejoy Doctor Electronics & Instruments Co. ,LTD Building 2, No.202 Zhenzhong Road -  
West Lake Economy & TechnologyZone 310030 Hangzhou China

SHANGHAI INTERNATIONAL TRADING CORP. GMBH Eiffestrasse 80, 20537 Hamburg, Germany

**Distribuito da / Distributed by:**  
MORETTI S.P.A.

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 2111

www.morettispa.com email: info@morettispa.com

MADE in P.R.C.

Distribuito da / Distributed by:

Via Bruxelles, 3 - Meleto  
52022 Cavriglia (Arezzo)  
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200  
[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)