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Venduto a _____

Via _____ Località _____

Hangzhou Sejoy Doctor Electronics & Instruments Co., LTD Building 2, No.202 Zhenzhong Road -
West Lake Economy & Technology Zone 310030 Hangzhou China

SHANGHAI INTERNATIONAL TRADING CORP. GMBH Eiffestrasse 80, 20537 Hamburg, Germany

Distribuito da / Distributed by:

MORETTI S.P.A.

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE in P.R.C.

INDEX

CODE.....	pag.30
INTRODUCTION.....	pag.30
INTENDED USE.....	pag.31
REGULATIONS AND DIRECTIVE OF REFERENCE.....	pag.31
GENERAL WARNING.....	pag.31
SYMBOLGY.....	pag.32
GENERAL DESCRIPTION.....	pag.32
- Parts view.....	pag.34
- Graphic symbols on display.....	pag.36
OPERATING THE UNIT AND CONNECTING PARTS.....	pag.38
- Installing batteries.....	pag.38
BEFORE USE.....	pag.38
CAUTIONS BEFORE USE.....	pag.39
HOW TO USE.....	pag.40
- Quick start test.....	pag.40
- Time/Date Setting.....	pag.40
- Memory group icon settings.....	pag.41
- Reading Memories.....	pag.41

- Last 3 tests average.....	pag.41
- Memory deletion.....	pag.41
- Pressure measurement.....	pag.23
MAINTENANCE.....	pag.45
CLEANING AND DISINFECTION.....	pag.45
CONDITIONS OF DISPOSAL.....	pag.46
Treatment of batteries.....	pag.46
Disposal instructions of waste batteries.....	pag.46
TROUBLESHOOTING.....	pag.47
SPARE PARTS AND ACCESSORIES.....	pag.48
SPECIFICATIONS.....	pag.48
WARRANTY.....	pag.50

Class I medical and measuring device Legislative
decree 24/02/97 n.46 in accordance with EEC
directive 93/42 and its amendments

CODE

DM480-Digital sphygmomanometer wrist type, LCD display 2.5"

INTRODUCTION

Thank you for purchasing LOGIKO sphygmomanometer by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

In case of doubts, we recommend to contact the dealer who can help and advice you properly

WARNING!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

INTENDED USE

The digital sphygmomanometer wrist type is measurement device for systolic and diastolic blood pressure

REGULATIONS AND DIRECTIVE OF REFERENCE

The LOGIKO sphygmomanometer by Moretti S.p.A are in accordance to the following regulations:

EN 1060-1 Non-invasive blood pressure measuring equipment
General requirements

EN 1060-3 Non-invasive blood pressure measuring equipment
Supplementary requirements for electromechanical blood pressure measuring systems

GENERAL WARNINGS

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment, otherwise injury or damage may occur
- Advise with physician or therapist to determinate the correct adjustment and the correct use of the sphygmomanometer
- Keep the packed sphygmomanometer away from heat sources
- SERVICE LIFE- the wheelchair use limit is defined by the wear of the parts
- DO NOT allow children to play on or operate with the sphygmomanometer

SYMBOLOLOGY

 serial number	 warning	 ECREP	 type B	 instruction
 Lot	 date of production	 disposal	 CE mark	 manufacturer

GENERAL DESCRIPTION

What is Blood Pressure?

Blood pressure is the pressure exerted on the artery tube while blood flows through the arteries. The pressure measured when the heart contracts and sends blood out of the heart is systolic (highest) blood pressure. The pressure measured when the heart dilates with blood flowing back into the heart is called diastolic (lowest) blood pressure. Blood pressure is measured in mmHg. It is submitted to many fluctuations and it depends on many elements. It is usually lower in summer and higher in winter. Blood pressure can change according to atmospheric pressure and it is affected by physical efforts, temperament, stress, meals, medicines, alcohol, smoking, age..etc...


It is recommended to note down your daily measurements and to consult with your physician who will tell you your normal blood pressure range. For the above reasons do not worry if you encounter some measurements at high level, consult with

your physician who will tell you the point at which you will be considered at risk.

Why measure your blood pressure?

Among the various health problems afflicting modern people, those associated with blood pressure are by far the most common. High blood pressure's dangerously strong correlation with cardiovascular diseases and high morbidity has made measuring blood pressure as a necessity in identifying those at risk.


NOTE Blood pressure can change with the age, it is necessary to consult with a physician to know your normal blood pressure range. Never change drug dosage without consulting with the physician!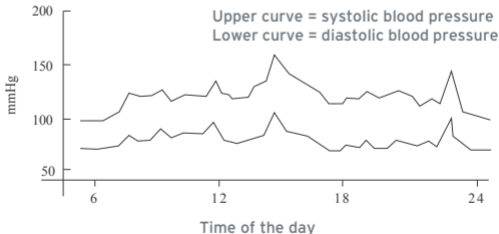


Example: Fluctuation within a day (Male 35 years old)

The following scheme has been developed by The World Health Organization (WHO), and it identifies the low- and high-risk blood pressure areas. This standard, however, is a general guideline as individuals' blood pressure varies among different people and different age groups, it is always subjective.

Parts view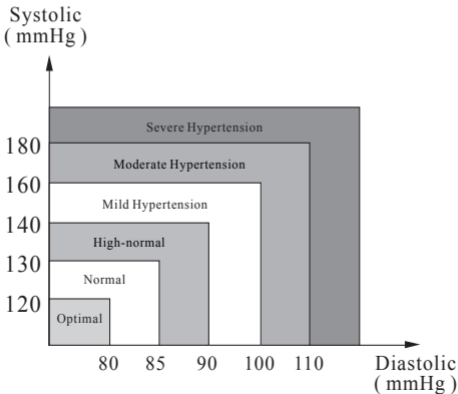


Contents

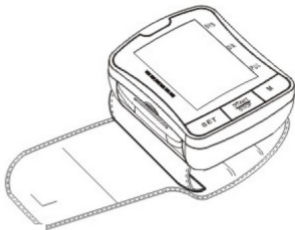Monitor Unit


Owner's Manual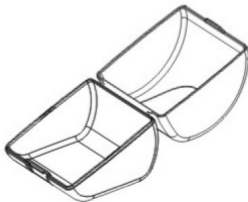



Plastic Storage Case

Graphic symbols on display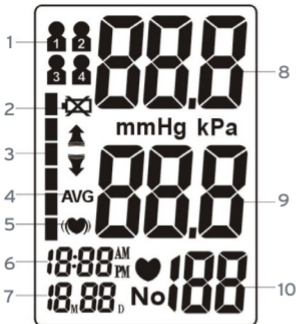

1. Groups of memories (1/2) only mod.
2. Low battery indicator
3. WHO blood pressure classification indicator
4. Last 3 measurements average
5. Irregular heartbeat indicator
6. Time
7. Date
8. Systolic blood pressure
9. Diastolic blood pressure
10. Pulse rate/min

NOTE The arrow indicator will show your current measurement value according to WHO classification. See the scale here below: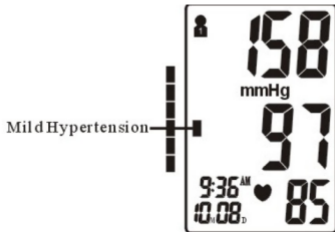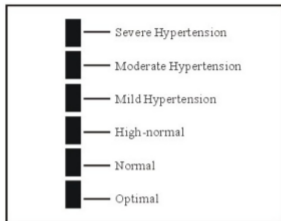


■: Blood Pressure Classification Indicator

OPERATING THE UNIT AND CONNECTING PARTS

Installing batteries

The battery compartment is situated in the back of the device. Press down slightly and let slide the cover in the arrow direction.

Install 2 AAA alkaline 1,5V batteries according to the indications inside the compartment. Replace the cover of the battery compartment.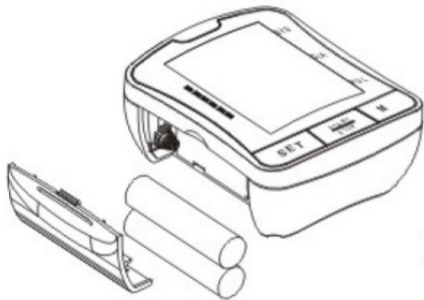

BEFORE USE

- Check if the device is damaged in order to guarantee a safe use for people and things
- Check the correct assembly, paying particular attention to correctly placing batteries and armcuff
- Verify the batteries correctly works, in case substitute them

CAUTIONS BEFORE USE

- Avoid any type of physical exercise for at least 30 min before the measurement
- Sit in a calm environment for at least 5 minutes before the measurement
- Avoid speaking during the measurement
- Do not move during the measurement
- Do not stand during the measurement. Sit in a relaxed position, rest your elbow and be sure the cuff is at heart level
- During the measurement, avoid electromagnetic interference with other electronic devices
- Wait 3 min before a new measurement
- If you need to measure your blood pressure every day, do it at the same time of the day
- Results comparisons should be made only when the monitor is used on the same arm, in the same position and at the same time of the day
- This blood pressure monitor is not recommended for people with strong arrhythmia.
- Do not use the device when damage and/or malfunction has occurred
- Remove batteries when the unit is not in use for an extended period of time

HOW TO USE

NOTE install batteries to use digital sphygmo-
manometer (see “BEFORE USE” section)

Quick start test

Press START/STOP button and keep pressed until you hear “beep”. The LCD display turns on for a second for a self-test.

At the end the display will show 0 for both, SYS and DIA pressure and this means the device is ready for use.

NOTE If there’s still some pressure in the cuff, the indicator in low will flash. Deflate completely the cuff.

Time/Date settings

- When power off, press “START/STOP” button for 3 seconds to set the Time/Date mode.
- “M” (stands for month) is flashing
- Press “MEM” button for set up
- Press “START/STOP” button to confirm it
- “D” (stands for day) is flashing
- Press “MEM” button for set up
- Set the day (“D”), hour (in AM/PM format) and minute in the same way.
- Press “START/STOP” button for 3 seconds to turn the unit off. All information will be saved

Memory group icon settings

- When power off, press "SET" button to start settings
- The Memory Group icon flashes
- Press "M" button to choose a group settings (group 1 or 2)
- Press "START/STOP" button to turn the unit off. All information will be saved

Reading Memories

- When power off, press "M-MEM" button
- Latest test result group will be shown (see Memory Group settings)
- Press " M-MEM" button to show different test results (up to 60 memories per group)

Last 3 tests average

- It allows checking the average test result from the last 3 readings of the last group used (AVG symbol will appear on display)
- When power off, press "M" button to activate screen display

Memory deletion

- Enter the memory reading mode (please see paragraph "Reading Memories").
- When memories are displayed, push and hold for 3 seconds START/STOP
- Memories are erased and the display shows "-- --"
- Push START/STOP to switch the device off

NOTE this procedure must be independently applied on both memories

Pressure measurement


1. The place and surroundings should be warm and quite.
2. Sit in front of a table or desk, where you can easily rest your arm

3. Wait some minutes before start measuring, in order to make pulse and blood flow stabilize.

4. Remove any garment from left arm, where cuff will be worn.

NOTE Be sure that any piece of clothing do not tighten the arm where pressure measurement is taken.

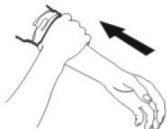5. Wrap the cuff around the naked wrist.

6. Lay the elbow on the table, so that keeps the same level of heart, with the palm of hand facing up.

7. Push START/STOP button to start measurement (the device will switch on; after an initial test making the display show all the characters, the cuff will automatically inflate).

NOTE The cuff's initial inflation pressure is 190 mmHg. If the patient's Systolic pressure is higher than the cuff's initial inflation pressure, the device will automatically further inflate the cuff.

8. When the cuff is inflated, the internal compressor will stop and the pressure measurement will start, slowly deflating the cuff (during deflating, the symbol  will blink on the display to show heart rate).

9. Stand totally still during the whole pressure measurement.

10. When the measurement is finished, the following results will be shown on the display:

- Systolic pressure (SYS)
- Diastolic pressure (DIA)
- Beats per minute (PULSE)
- Classification of values according to standard WHO/OMS (World Health Organization)
- Indicator of irregular beats (Arrhythmia) ()

NOTE This symbol will be shown only if the device detects two or more irregular beats during measurement.

MAINTENANCE


The LOGIKO devices by Moretti S.p.A are checked carefully and supplied with EC mark, once launched on the market.

For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair, use only original spare parts and accessories.

CLEANING AND DISINFECTION

- **DO NOT immerse the device!**
- **Cleaning:** use only a damp cloth. Pay attention to the display
- **Disinfection:** use a cleanser, avoiding the liquid seep into the device

NOTE Never use acids, alkalis or solvents such as acetone

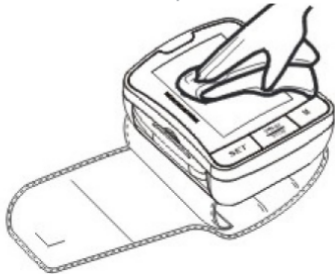CONDITIONS OF DISPOSAL

-General conditions of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the sphygmomanometer in the appropriate disposal areas for recycling.

TREATMENT OF BATTERIES (Directive 2002/96/EC)

At the end of its life, the product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment or at retailers that provide this service. By ensuring these batteries are disposed of correctly, you will help prevent potentially negative consequences for the environment and human health which could otherwise be caused by inappropriate waste handling of the battery. To remark the need to dispose of electrical equipment separately, the products is marked with crossed mobile waste bin

DISPOSAL OF WASTE BATTERIES - Directive 2006/66/EC:

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TROUBLESHOOTING

PROBLEM	POSSIBLE CAUSE	SOLUTION
Blood pressure results are not reliable	<ol style="list-style-type: none"> 1. Cuff is too tight or not properly positioned on the arm 2. Body or device movements during measurement 	<ol style="list-style-type: none"> 1. Firmly reposition cuff as illustrated in point 5, paragraph Pressure Measurement 2. Stand still and relaxed during measurement, avoiding speaking
"ERR" displayed	<ol style="list-style-type: none"> 1. Cuff fails to inflate properly 2. Improper operation 3. Pressurization is over 300mmHg 	<ol style="list-style-type: none"> 1. Check connections between cuff and device 2. Read user manual carefully and do the operation properly 3. Read user manual carefully and do the operation properly

SPARE PARTS AND ACCESORIES

For spare parts and accesories refer to main catalogue

SPECIFICATIONS

PRODUCT DESCRIPTION	Wrist type fully Automatic Blood Pressure monitor
MODEL	DM480
LCD DIGITAL DISPLAY	49 x 38 mm (2,5")
MEASUREMENT METHOD	Oscillometric method
MEASUREMENT RANGE	Pressure 0 mmHg / 300 mmHg Pulse 30 to 180 Beats/Minute
MEASUREMENT ACCURACY	PRESSURE 3 mmHg PULSE 5%
PRESSURIZATION	AUTOMATIC PRESSURIZATION
MEMORY	120 memories in two groups with date and time
FUNCTION	Irregular beat detection (arrhythmia) WHO/OMS classification indicator Last 3 result average - AVG Automatic Power off

POWER SOURCE	2 AAA batteries - 1.5V
BATTERY LIFE	APPROXIMATELY 2 MONTHS AT 3 TESTS PER DAY
UNIT WEIGHT	Approx. 120 g (excluding batteries)
UNIT DIMENSIONS	77 x 64 x 32,5 mm
CUFF CIRCUMFERENCE	13,5 cm - 21,5 cm
OPERATING ENVIRONMENT	Temperature 5°C - 40°C Humidity < 85%RH Pressure Atmospheric Pressure
STORAGE ENVIRONMENT	Temperature 10°C - 55°C Humidity < 95% RH Pressure 525mmHg - 795mmHg
CLASSIFICATION	-Internal Powered Equipment. Type B -Cuff is the Applied Part

WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to.

The correct intended use is specified in this manual.

Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

RECHARGEABLE BATTERIES WARRANTY

The original batteries and the spare batteries are covered by a 90 days warranty during a period of 6 month, with regards to manufacturing defeat or to what concern the law regulation. If charge batteries are unused for more than 3 consecutive months, the warranty is void. If exhausted batteries are unused for more than 3 consecutive days, the warranty is void.

REPAIRING

WARRANTY REPAIR

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

REPAIR NOT COVERED BY WARRANTY

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

EXEMPT CLAUSES

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, , including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

WARRANTY CERTIFICATE

Product _____

Purchasing date _____

Authorized dealer _____

Street _____ Place _____

Sold to _____

Street _____ Place _____

 Hangzhou Sejoy Doctor Electronics & Instruments Co., LTD Building 2, No.202 Zhenzhong Road -
West Lake Economy & Technology Zone 310030 Hangzhou China

 SHANGHAI INTERNATIONAL TRADING CORP. GMBH Eiffestrasse 80, 20537 Hamburg, Germany

Distribuito da / Distributed by:

MORETTI S.P.A.

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.

Distribuito da / Distributed by:

Via Bruxelles, 3 - Melegnano
52022 Cavriglia (Arezzo)
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200
www.morettispa.com
info@morettispa.com