CE

**EASYRED®**

MA EMIBO 01A

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## **DISASTER POUCH INSTRUCTION MANUAL**

**MORETTI S.P.A.**  
Via Bruxelles 3 - Melegnano  
52022 Cavriglia (Arezzo)  
Tel. +39 055 96 21 11

Fax +39 055 96 21 200  
[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)

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I Class Medical Device  
according EEC directive 93/42

## Code EM180

Thank you for purchasing the **EASYRED** by Moretti disaster pouch. This product is made of PVC and it is disposable.  
Read this instruction manual before the use.

### Intended use

Device for corpse transfer.

### Technical features

Size: 210x 90cm

Max loading weight: 150 kg.

### Instructions

1. Open the pouch by internal and external zipper.
2. Lay the corpse on the pouch and close the zipper.
3. Move the corpse.

### Warning

1. Pay attention to the transmission of infectious diseases during the corpse transport.
2. Only qualified personnel can use this product and can assist the injured person; pay attention, above all, in case of spinal lesions.
3. Do not use this product for purposes not indicated in this manual.
4. Moretti SpA is not responsible for purposes not indicated in this manual.

### Maintenance and Cleaning

This product is disposable; keep far from heat sources.

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## WARRANTY

This disaster pouch has 2-YEAR warranty from purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals. In case you need to get the product repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

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