CE

EASYRED®

EM210

EM215

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VACUUM MATTRESSES INSTRUCTION MANUAL

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I Class Medical Device
according EEC directive 93/42

Codes EM210 - EM215

Thank you for purchasing the **EASYRED** by Moretti vacuum mattress. This product is realized for patient immobilization during transport. The mattress is x-ray translucent.

Read this instruction manual before the use.

Intended use

Device for rescue

Technical features

EM210

Size: 200x95cm

weight: 8.1kg

Max loading weight: 150 kg

Material used: vinyl and nylon

EM215

Size: 210x90cm

weight: 9.8kg

Max loading weight: 150 kg

Material used: vinyl and polyester

Supplied with 4 safety belts

Instructions

1. Turn the valve to the position of "open".
2. Lay the mattress down
3. Insert the pump in the valve and suck up the air until the mattress becomes semi-rigid. Then turn the valve in the position of "closed".
4. Transfer the patient on the mattress and immobilize him with safety belts.
5. Turn the valve to the position of "open" again, then take out the air until the mattress turns enough hard to immobilize the patient. Then, turn the valve to "close" and pull out the pipe of the pump.
6. There are several handles around the mattress which can be used to transfer the patient onto the ambulance or stretcher.

Warning

1. If you find an air leak during the process, check carefully if the valve is tightly closed.
2. The mattress does not have to be in contact with sharp things.
3. Only qualified personnel can use this product and can assist the injured person; pay attention, above all, in case of spinal lesions.
4. Do not use this product for purposes not indicated in this manual.
5. Moretti SpA is not responsible for purposes not indicated in this manual.

Maintenance and Cleaning

Use water and non aggressive disinfectant; dry far from sun rays.

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WARRANTY

These mattresses has 2-YEAR warranty from purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals. In case you need to get the product repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

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Acquistato in data / Purchasing date _____

Rivenditore / Authorized dealer _____

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Venduto a / Sold to

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MORETTI S.P.A.

www.morettispa.com - email: info@morettispa.com

Made in P.R.C.