- Manuale di istruzioni
- Instruction manual
- Manuel d'instruction
- Manual de instrucciones

Moretti S.p.A.
Via Bruxelles 3 - Meleto
52022 Cavriglia (Arezzo)
Telefono +39 055 96 21 11

Fax +39 055 96 21 200
www.morettispa.com
info@morettispa.com

KYARA

ON/OFF

ITALIANO

ENGLISH

FRANCAIS

ESPAÑOL

Puffy

NEBULIZATION

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ITALIANO

KYARA

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KYARA

INTRODUCTION

Your **PUFFY** Ultrasonic should be used the supervision of a licensed physician and/or a respiratory therapist.

Thank you for purchasing a **PUFFY** Ultrasonic.

With proper care and use, your Nebulizer will provide you with many years of reliable treatments.

Your **PUFFY** Ultrasonic has been designed to offer compact, portable and lightweight (under 350g) convenience while receiving inhalation treatments. This unit requires ordinary tap or distilled water and your physician-prescribed medication to create an effective high frequency mist. Treatments can be administered discreetly thanks to advanced ultrasonic technology making your Nebulizer virtually silent during operation.

An easy airflow volume control of treatment levels as recommended by your physician and/or respiratory therapist. This unit operates on standard AC power or an optional battery pack. Treatments are delivered quickly, safely and conveniently making this unit ideal for all ages.

We encourage you to thoroughly read this guidebook to learn about the features of your **PUFFY** Ultrasonic.


Together with your physician and/or respiratory therapist, you can feel comfortable and confident knowing that you are obtaining the most effective inhalation treatments for your respiratory condition.

INDEX

Introduction	2
Product Identification	3
Cautions	4
Operating Your Nebulizer	4
Cleaning Procedures	5-6
Troubleshooting	8
Rechargeable Battery	8-9
Declaration of conformity CE	11
Warranty.....	12

The **PUFFY** nebulizer is indicated for the treatment of asthma and other respiratory problems for which the aerosol therapy is recommended. The **PUFFY** device is not to be used with "Pentamidine". This nebulizer is indicated for an individual use.

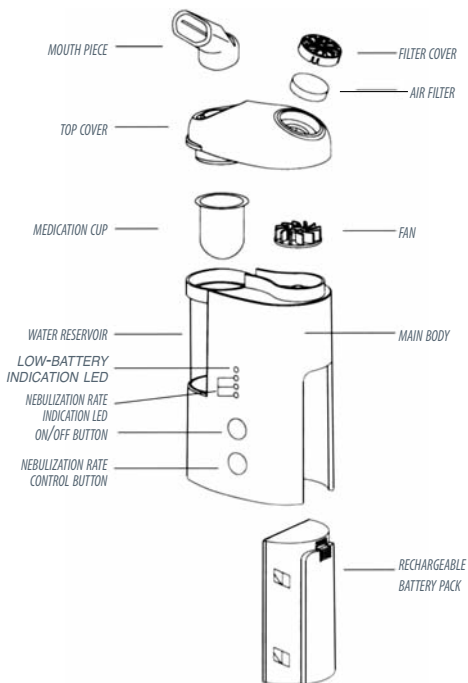PRODUCT IDENTIFICATION

CAUTIONS PRODUCT CAUTIONS

Please use general safety precautions when operating your **PUFFY**.

This unit should be used only for (1) its intended purpose as described in this guidebook, and (2) with medications only under the supervision and instruction of your physician.

READ THE FOLLOWING BEFORE USING

- To avoid electrical shock: Keep unit away from water • Do not handle the unit or power cord with wet hands • Do not immerse the AC adapter or unit in liquid (Do not use while bathing • Do not reach for a unit that has fallen into water-immediately unplug the unit.
- Do not use the unit if it has any damaged parts (including power cord or plug), if it has been submersed in water or dropped.
Promptly send the unit for examination and repair. • The unit should not be used where flammable gas, oxygen or aerosol spray products are being used.
- Keep the air vents open. Do not place the unit on a soft surface where the openings can be blocked.
- If the water reservoir or medicine cup is empty, do not attempt to operate the unit.
- If any abnormality occurs, discontinue use until the unit has been examined and repaired.
- The unit should not be left unattended while plugged in.
- Do not use solutions other than water in the tank. It may damage the transducer.
- Do not tilt or shake the unit when in operation.
- Opening the top cover during operation may cause a severe injury.
- Disconnect the unit from the electrical outlet before cleaning filling and after each use.
- Do not use attachments unless recommended by the manufacturer.
- The unit should not be used for more than 30 minutes at a time.
- Wait 30 minutes before reusing the unit.

OPERATING CAUTIONS

- Keep your eyes away from the output of medication mist.
- Change the water after each use.
- The maximum capacity of the medicine cup is 8 ml. And should not be overfilled.
- Dilute dense medications or medication volumes 2 ml. or less with

- distilled water or saline solution to obtain a volume of at least 4 ml.
- Do not use this unit while operating a vehicle.
 - If any discomfort or abnormality occurs, stop using the unit immediately.

Storage Cautions

- Do not store the unit in direct sunlight, high temperature or humidity.
- Keep the unit out of reach of small children.
- Always keep the unit unplugged while not in use.

Cleaning Cautions

- Check the medication cup, mouth nozzle and air filter before each use. Dirty or worn parts should be replaced.
- Do not immerse the unit in water. It may damage the unit.
- Disconnect the unit from the electrical outlet before cleaning.
- Clean all necessary parts after each use as instructed in this guidebook.

CLEANING PROCEDURES

It is important to clean your **PUFFY** before the first time it is used and after each use. Remember: Disconnect the unit from the electrical outlet and never immerse the unit in water.

Cleaning the parts:

The mouth nozzle, nose piece, extension tube, mask, top, cover, medicine cup, airflow cover, fan and air filter should be cleaned with a mild detergent or commercially available disinfectant. Thoroughly rinse each part and allow to completely air dry before using.

Cleaning the main unit:

The main unit should be cleaned with a soft cloth moistened with a non-abrasive cleanser and water.

OPERATING YOUR NEBULIZER

Follow the previous cleaning instructions prior to using your **PUFFY** Ultrasonic for the first time or after it has been stored for an extended period.

Remember: Always unplug this product and turn off the switch at bottom before cleaning and filling and after each use.

To fill the water tank,

1. Remove the top cover by pulling straight up, Fig. 1. Remove the medicine cup if applicable.

WATER BETWEEN
THESE TWO LINES

2. Select either tap or distilled water* that has a temperature range of 20° F - 30° F , Fig. 2. * Distilled water is preferred.

3. Fill the water reservoir with tap or distilled water to the indicated level (between the 2 lines located inside the water reservoir), Fig. 3. Remember: Do not use solutions other than water in the reservoir. It may damage the transducer - Fill to the correct level.

Note: • The use of cool water (below 68°F or 20°C) will result in a lower rate of nebulization. • The use of warm water (above 86°F or 30°C) will result in a higher rate of nebulization.

To fill the medicine cup,

1. Place the medicine cup inside the water reservoir.
2. Pour the medicine directly into the medicine cup.
3. Replace the top cover.

Remember:

- The maximum capacity of the medicine cup is 8 ml. and should not be overfilled.
- Dilute dense medications or medication volumes 2 ml. or less with distilled water or saline solution to obtain a volume of at least 4 ml.
- Only use medications under the instructions of your physician.

To connect the AC adapter,

1. Insert the adapter plug into the unit's AC port, Fig.4.
2. Plug the adapter into a standard AC outlet.

To start the unit,

1. Place the unit on a solid, flat, dry surface making sure that the water reservoir and medicine cup have been properly filled and the mouth nozzle is properly attached to the top cover.

2. Turn the switch at bottom of unit to "ON" position The LED

indicator should be lit.

3. Press the ON/OFF button.

Note:

- Your unit has been designed to automatically output the highest level of airflow.
- The Airflow Volume Indicator Light will illuminate at the highest setting.

To adjust the airflow, and nebulizer rate

1. Your Ultrasonic Nebulizer has 3 airflow setting: Maximum, Medium and Minimum.

2. Based on instructions from your physician and/or respiratory therapist, press the Nebulization rate control button to adjust the airflow volume.

The Airflow volume Indicator Light will show the lever (Maximal, medium or Minimum) selected, *Fig. 5.*

FIG. 5

To begin a treatment,

1. Follow the instructions provided by your physician and/or respiratory therapist.

2. Your Nebulizer is designed to start at a lower atomization rate in the beginning and will become stronger after a few seconds.

Remember:

- *Keep your eyes away from the output of medication mist.*
- *If any discomfort or abnormality occurs, stop using the unit immediately.*
- *Do not tilt or shake the unit when in operation.*
- *Opening the top cover during operation may cause a severe injury.*

To end a treatment,

1. When the treatment has ended, press the ON/OFF button stop operation. Turn the switch at bottom to "OFF/CHARGE" position. Otherwise, your unit will automatically shut off after approximately 10 minutes of continuous use.

2. Unplug the adapter from the outlet and detach the adapter plug from the unit's AC port.

3. Allow the unit to cool - approximately 20-30 minutes.

Remember:

- *The unit should not be used for more than 30 minutes at a time.*
- *Wait 30 minutes before reusing the unit.*

After every use,

1. Allow the unit to completely cool.
2. Remove the top cover.
3. Carefully remove the medicine cup and pour out remaining medication.
4. Empty the water reservoir.
5. Wipe out the water reservoir and transducer with a soft, lint-free cloth.
6. Follow the cleaning procedures provided in this guidebook.

TROUBLESHOOTING

If any abnormality occurs during use, please check and correct the following:

Problem:

Unit does not operate when ON/OFF button is pressed.

Solution:

Check the AC adapter connection to the outlet and/or check the AC plug to the unit's AC port.

Check the switch at bottom is turned on.

Check that the water reservoir is filled to the correct level.

If using with optional battery pack, check that the battery is completely charged. Check that the battery is mounted correctly to the unit.

Problem:

No misting or low rate of misting

Correction:

Check that there is medication in the medicine cup.

Check the medication cup for damage.

Check that the transducer is clean.

Problem:

Fan is not working properly.

Correction:

Check that the fan is firmly and properly mounted on the motor shaft.

REPLACEMENT PARTS

Replacement parts are available through your local retailer or they can be ordered by mail. Your **PUFFY** Ultrasonic can operate with the use of a rechargeable battery when AC power is not available. The life of the battery is approximately 2 years. However, battery life will vary depending on usage, storage and charging conditions.

To charge the battery,

1. Install the battery pack by sliding it into the right-bottom of nebulizer.
2. Make sure the switch at bottom is turned to "OFF/CHARGE" position.
3. Connect adapter's plug to nebulizer's power receptacle.
4. Plug the adapter into an AC outlet. *Fig. 1.*

Note:

- For first time use or after storing the battery for a period of time, charge the battery at least 24 hours, and thereafter 4 to 12 hours depending on the length of usage. • A fully charged battery will last for 20-30 minutes of continuous use. • To prolong the life of the battery, the charging current is designed at the minimum level. Therefore, it is recommended to charge the battery after each use and let it charge overnight.

FIG2

To use the battery,

FIG. 2

1. Install the battery by sliding it into the right bottom of nebulizer. *Fig. 2*
2. Begin to use the Nebulizer as described in the operating Your Nebulizer section of this guidebook.
3. If the nebulizer's indication led changes to red color, it indicates that the battery is in low voltage condition and needs to be charged.
4. After use, dispose excess of medication and water before disconnecting the battery.

Battery Caution

- Do not use the battery with equipment other than your **PUFFY** Ultrasonic, or attempt to modify it for other use.
- Do not store the battery in direct sunlight, high humidity.
- Avoid placing the battery on metal or wet surfaces or inside a metal box. This may short-circuit the battery's positive and negative contact causing the battery to heat up or explode.

Unit Specifications

<i>Name and Model Number</i>	PUFFY Ultrasonic NB-08L
<i>Power</i>	100-120VAC or 220-240VAC 50/60Hz
<i>Power Consumption</i>	Approximately 10W
<i>Ultrasonic Frequency</i>	2.5 MHz
<i>Nebulization</i>	3-speed adjustable 0.2 to 0.7 ml./minute depending on medicine density
<i>Medicine Cup Capacity</i>	Approximately 8ml. Maximum
<i>Timer</i>	Approximately 10 minutes
<i>Dimensions</i>	8.8 x 5.3 14.3cm
<i>Weight</i>	Approximately 230g. (350g. with battery pack)
<i>Accessories</i>	AC adapter, Adult & child masks, Extension tube, Carrying Case, 5 Reusable Medicine cups Rechargeable battery (optional)

Optional Battery Specifications

Name and Model Number	PUFFY Ultrasonic Rechargeable Battery RB-08L
Battery Type	Ni-MH
Battery Voltage	12 VDC
Output Voltage	12 VDC
Charging Voltage.....	12 VDC
Charging Current.....	40-100mA
Charging Time	Minimum 24 Hours first time; 4-12 Hours thereafter depending on usage
Capacity	480 mAh/ VDC (20-30 minutes continuous use)
Operating Temperature	0°C - 45°C
Storage Temperature	-18°C - 45°C (0~112°F)
Dimensions	4.7 x 2.8 x 8.4cm
Weight	Approximately 125g.

Specifications subject to change without notice

DECLARATION OF CONFORMITY CE

We, Moretti SpA, hereby declare under our sole responsibility that the products taking part of the aerosol therapy devices and produced by the company VEGA Technologies Inc. (Taiwan) classified as Class II. Devices comply with applicable requirements of the directive 93/42/EEC, annex II.

CE
0434

WARRANTY

This device has 2-year warranty from the purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge.

The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals. In the case you need to get your device repaired and you fulfil the conditions of warranty, send the device to the nearest retailer together with the purchase receipt.

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**MORETTI S.p.A. Via Bruxelles 3