KYARA®

REF LT594 / E08Z6

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IMPORTANT INSTRUCTIONS KEEP FOR FUTURE USE.

ATTENTION!

Instructions and warnings for a safe use.

Please read the operating instructions and safety warnings in particular before using the appliance and follow these at all times. Keep this manual and illustrative guide with the appliance for future consultation. Should you pass the appliance on to another user, make sure to also include this documentation. Should certain parts of this user manual be difficult to understand or should doubts arise, contact the company before using the product, at the address indicated on the last page.

CONTENTS

SAFETY WARNINGS	page 6
SYMBOLS	page 8
DESCRIPTION OF THE APPLIANCE	page 8
USING THE HEATING PAD	page 8
USE	page 8
SWITCHING THE APPLIANCE ON AND	
SELECTING THE TEMPERATURE	page 9
AUTOMATIC SHUTDOWN	page 9
SWITCH-OFF	page 9
CLEANING AND MAINTENANCE	page 9
WASHING INSTRUCTIONS	page 9
DISPOSAL	page 10
ILLUSTRATIVE GUIDE	page I-II
TECHNICAL DATA	page II

SAFETY WARNINGS

- After having removed the appliance from its packaging, verify its integrity based on the drawing and check for any damage caused during transport. If in doubt, do not use the

appliance and contact the authorised service centre.

- **Do not let children play with the packaging! Keep the plastic wrapping out of the reach of children; risk of suffocation!**
- **Before connecting the appliance, check that the mains voltage data reported on the identification plate corresponds to that of the available electric mains. The identification plate is attached to the appliance.**
- **This appliance is intended exclusively as a heating pad for home use. Any other use is improper and potentially dangerous.**
- **The appliance is not to be used by persons (including children) with reduced physical, sensorial or mental abilities, or with no experience or knowledge, unless supervised by a person responsible for their safety, supervision or after having received instructions regarding its use.**
- **Children must be supervised to ensure they do not play with the appliance.**
- **Do not use the appliance for ill people, for infants or for persons who are insensitive to heat.**
- **This appliance must not be used by children if the control devices have not been set beforehand by a parent or guardian or if the child has not been properly instructed on how to use the device safely.**
-  **WARNING – NOT suitable for children under the age of 3. Children under the age of 3 must not use the appliance as they are unable to react in the event of overheating.**
-  **DO NOT activate the temperature control with wet hands.**
-  **DO NOT pull the power cord or the appliance itself to remove the plug from the socket.**
- **Always disconnect the appliance from the mains before cleaning or performing maintenance and when the appliance is not in use.**
- **In case of a fault or malfunctioning of the appliance, switch it off and do not tamper with it. If the appliance needs to be repaired, contact an authorised technical service centre.**
- **If the power cord is damaged, it must be replaced by an authorised technical service centre, so as to prevent all risks.**
- **The appliance must be used solely with the types of control**

GB

indicated on the label.

- **Inspect the appliance frequently to check for signs of wear and tear. If such signs are found or if the appliance has been used incorrectly, contact an authorised technical assistance centre before continuing to use it.**
- **Persons with pacemakers must not use the appliance.**
- **This appliance is not intended for medical use inside of hospitals.**
- **⊘ DO NOT use the appliance if it is wet.**
- **⊘ DO NOT use the appliance to keep animals warm.**
- **⊘ DO NOT leave the appliance running while you sleep.**
- **⊘ DO NOT INSERT PINS OR NEEDLES.**
- **Application of the appliance for a long time at high temperature can burn the skin.**

SYMBOLS

	Protection class II.		Prohibited actions.
	Warning.		Temperature selector.
	ON/OFF selector.		STAND-BY.

DESCRIPTION OF THE APPLIANCE

- | | | |
|------------------------|-------------------------|-------------------|
| 1 Temperature display. | 3 Temperature selector. | 5 Control socket. |
| 2 Stand-by Display. | 4 ON/OFF selector. | 6 Plug. |

USING THE HEATING PAD

The heating pad is designed to warm the human body. It allows for targeted application of heat. The heating pad can be placed on parts of the body, such as: neck, shoulders, back, abdomen and joints, depending on the relative model.

USE

CONNECTING AND REMOVING THE TEMPERATURE ADJUSTMENT CONTROL

ATTENTION!

The heating pad can only be used with the control supplied. The relative model is specified on the label sewn on the panel.

- Insert the control socket (5) into the socket (6) to connect the temperature adjustment control to the heating pad.
- Insert the plug in the socket.
- Place the heating pad on the part of the body that is to be heated.

SWITCHING THE APPLIANCE ON AND SELECTING THE TEMPERATURE

- Press the ON/OFF  (4) selector to switch the heating pad on.
- The display (1) will light up and the temperature will be automatically set to position 5.
- Select the desired temperature (from 1 to 5) by pressing the temperature selector  (3) until the relative number lights up on the temperature display (1).

AUTOMATIC SHUTDOWN

- The heating pad is provided with an automatic switch-off system. The stand-by display  (2) lights up after 3 operating hours to indicate that the product has switched off automatically.
- Switch the appliance back on by pressing the ON/OFF selector (4) and select the desired temperature once again.

SWITCH-OFF

- Switch the heating pad off by pressing the ON/OFF selector (4) when it is no longer used.
- Remove the plug from the socket.

CLEANING AND MAINTENANCE

The heating pad is washable. Refer to the washing symbols on the heating pad label for the washing instructions.

 Wash at 30°C.	 Do not bleach.
 Tumble-dry at less than 40°C.	 Do not iron.
 Do not dry-clean.	

WASHING INSTRUCTIONS

Always disconnect the plug and the control socket (5) from the socket (6) before cleaning. Slightly push down the plug to remove it easily

 **MACHINE WASHING**

- The heating pad can be machine washed at 30°C with a normal cycle.
- Only the gentle spin cycle is to be used. DO NOT set an additional spin cycle.

 **DRYING**

- Set the dryer at a low temperature.
- Do not apply pegs on the electrical parts when hanging the heating pad to dry in order not to damage them.
- DO NOT use the heating pad until it is completely dry.



DISPOSAL

The product is packaged in recyclable materials. Dispose of it in compliance with the environmental protection standards.

The product must be disposed of appropriately, in compliance with European standard 2002/96/EC. The recyclable materials contained in the appliance must be recovered so as to prevent environmental pollution. For further information, contact the local disposal body or the appliance dealer.

M1001467
0114 (00YY)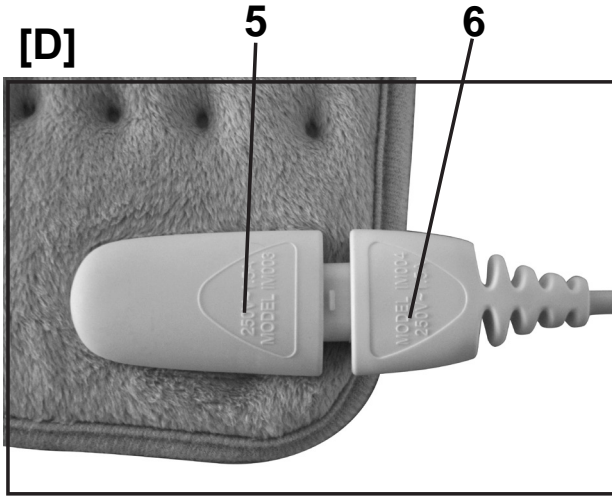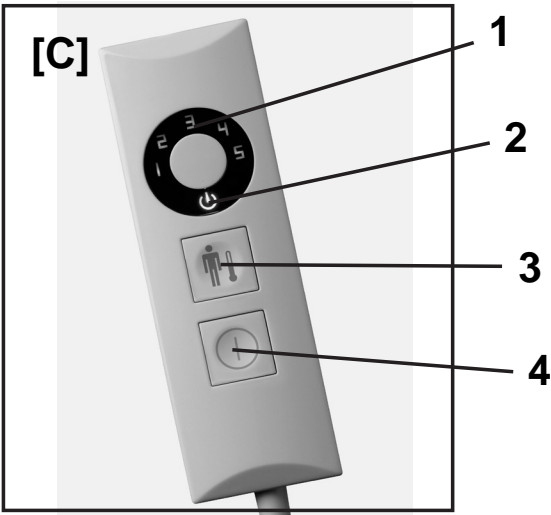

**DATI TECNICI
TECHNICAL DATA**

TYPE E08Z6 220-240V 50 Hz 15 W (max. 110 W)

Distribuito da / Distributed by:

Via Bruxelles 3 - Meleto
52022 Cavriglia (Arezzo)
Tel. +93 055 96 21 11

Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com