GRADO®

RM400

RM410

RM420

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 Hangzhou Sejoy Doctor Electronics & Instruments Co., LTD Building 2, No.202 Zhenzhong Road -
West Lake Economy & Technology Zone 310030 Hangzhou China
 SHANGHAI INTERNATIONAL TRADING CORP. GMBH Eifefstrasse 80, 20537 Hamburg, Germany

Distribuito da / Distributed by:
MORETTI S.P.A.

Via Bruxelles, 3 - Melegnano 20022 Cavriglia (Arezzo) Tel. +39 055 96 21 11
www.morettispa.com email: info@morettispa.com
MADE in P.R.C.

GRADO

English

**Class IIa medical and measuring device
in accordance with EEC directive
93/42 and its amendments**

CODE

- RM400-** Rigid digital thermometer, 60 sec.
- RM410-** Rigid digital thermometer, 30 sec.
- RM420-** Flex digital thermometer, 30 sec.

INTRODUCTION

Thank you for purchasing GRADO by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

In case of doubts, we recommend to contact the dealer who can help and advice you properly.

INTENDED USE

The thermometer is an instrument designed to measure body temperature

CAUTION!

- Do not use the product for a purpose not indicated in this manual
- **Moretti S.p.A** declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice

DECLARATION OF CONFORMITY

The GRADO thermometers by Moretti S.p.A complies with directive 93/42/EEC, medical devices and further modifications and complies with EN 12470-3: Clinical thermometers. Part 3: performance of compact electrical thermometers.

GENERAL WARNINGS


- **DO NOT** use this product or any available optional equipment without first completely reading and understanding this instruction manual.
- Keep the packed thermometer away from heat sources
- Do not store or keep in direct sunlight, or in high temperature
- Service life - the thermometer use limit is defined by the wear of the parts
- Do not allow children to play on or operate the thermometer
- Do not use in the ear

GENERAL DESCRIPTION

The digital thermometers provide a fast and accurate reading of the body temperature. They have been designed for rectal, oral and axillary measurements

The device bears the mark CE0197 according to the European regulation for Medical Devices. The device isn't subject to the calibration obligation though a quality control in accordance with directive 93/42/EEC

Parts view

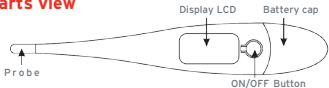HOW TO USE

- Do not bend, drop the thermometer
- Do not boil probe to disinfect
- Do not disassemble, except in case of battery replacement
- The thermometer should only be used under the supervision of an adult. Prohibit children from biting, hitting and removing the battery cover, dislodging and potentially swallowing the battery
- Do not use the device in case of strong electromagnetic field such as microwave oven or cell phone

BODY TEMPERATURE

Body temperature can vary from 36.1°C to 37.2°C. The generally accepted "normal" temperature is 37.0°C. Usually your temperature is lower in the morning than in the afternoon. Normal temperature varies according to the location of the temperature reading. While oral temperature follows the guidelines previously described, rectal temperature is generally 0.5°C higher. Conversely, axillary temperature will be 0.5°C lower

HOW TO USE

Measure the temperature

1. Press On/Off button
2. You will hear a beep and the screen will display 1888.5, then will display the last reading and then the autotest temperature
3. Finally the screen will display Lo°C and the symbol °C start flashing
4. The thermometer will start to measure the temperature as soon as it is exposed to body heat
5. The °C symbol will continue to flash until completion of the temperature taking process
6. After the measurement, the thermometer will emit a sequence of acoustic signals
7. Push On/Off button to turn the unit off. If you do not turn off the unit, it will switch off automatically

NOTE If the measured temperature is less than 32°C the screen will display Lo, if the measured temperature is higher than 42.9°C the screen will display Hi

Method of measurement

There are three methods of measuring body temperature. Slight temperature differences may occur depending on the method of measurement

1. Oral measurement: using the diagram, place the probe tip under the tongue as near as possible to the heat pocket. Close your mouth and breathe evenly through the nose to prevent the measurement from being influenced.

2. Axillary measurement: the thermometer should be placed under the arm so the probe tip is touching the skin. Hold the thermometer parallel to the body and position the arm on the chest so the tip is well covered by the arm. This insures the probe is not affected by the room's air.
3. Rectal measurement: rectal temperature should be taken when is impossible or impractical to take oral or axillary temperature. The probe should be inserted a maximum of 1-2 cm.

Replace the battery

1. The battery should be replaced when the screen display 
2. Pull battery cover off as shown in Fig.
3. Use pointed object such as a pen to remove old battery. Replace with new 1.5DC button type
4. Position the battery cover on the thermometer

MAINTENANCE

The GRADO devices by Moretti S.p.A are checked carefully and supplied with EC mark, once launched on the market.

For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years.

In case of repair, use only original spare parts and accessories.

CLEANING AND DISINFECTION

Cleaning use a damp cloth

Disinfection if you need to disinfect the device use isopropyl rubbing alcohol

NOTE Never use acids, alkalis or solvents such as acetone

CONDITIONS OF DISPOSAL

- General conditions of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the thermometer in the appropriate disposal areas for recycling.

TREATMENT OF BATTERIES (Directive 2002/96/EC)

At the end of its life, the product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment or at retailers that provide this service. By ensuring these batteries are disposed of correctly, you will help prevent potentially negative consequences for the environment and human health which could otherwise be caused by inappropriate waste handling of the battery. To remark the need to dispose of electrical equipment separately, the products is marked with crossed mobile waste bin the product bears the barred bin symbol.

DISPOSAL OF WASTE BATTERIES (Directive 2006/66/EC)

This symbol on the battery or on the packaging indicates that the battery provided with this product shall not be treated as household waste. By ensuring these batteries are disposed of correctly, you will help prevent potentially negative consequences for the environment and human health which could otherwise be caused by inappropriate waste handling of the battery. The recycling of the materials will help to conserve natural resources. At the end of their life hand the batteries over to the applicable collection points for the recycling of waste batteries. For more detailed information about recycling of this product or battery, please contact your local Civic Office, your household waste disposal service or the shop where you purchased the product.

TECHNICAL FEATURES

WEIGHT (INCLUDING
BATTERY)

RANGE

ACCURACY

MEMORY

BATTERY

BATTERY LIFE

DISPLAY

USING ENVIRONMENT

STORAGE CONDITION

CLASSIFICATION

DIMENSION

REGULATION

SIMBOLOGYV

 code	 Warning	 ECEP
 date of production	 disposal	 CE mark
 type B	 Lot	 manufacturer
		 instruction

11,8gr

32.0°C-42.0°C

±0.1°C, 35.5°C
 ±0.2°C under 35.5°C or over 42°C
 at standard room temperature of
 18°C to 28°C

Last measurement

One 1.5DC button type

APPROX 200 HOURS

3 DIGITS LCD

5°C to 40°C/ 85%RH

-10°C to 55°C/ 95%RH

Type B 

RM400
 12,3 x 1,9 x 1,1cm H
 RM410
 13,9 x 2,3 x 1,3 cm H
 RM420
 13,9 x 2,2 x 1,3cm H

EN 12470-3

WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use. Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

RECHARGEABLE BATTERIES WARRANTY

The original batteries and the spare batteries are covered by a 90 days warranty during a period of 6 month, with regards to manufacturing defeat or to what concern the law regulation. If charge batteries are unused for more than 3 consecutive months, the warranty is void. If exhausted batteries are unused for more than 3 consecutive days, the warranty is void.

REPAIRING

WARRANTY REPAIR

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

REPAIR NOT COVERED BY WARRANTY

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

NON-DEFECTIVE DEVICE

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be paid by the customer.

SPARE PARTS

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

EXEMPT CLAUSES

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

WARRANTY CERTIFICATE

Product _____ Purchasing date _____

Authorized dealer _____ Street _____ Place _____

Sold to _____ Street _____ place _____

 Hangzhou Sejoy Doctor Electronics & Instruments Co.,LTD Building 2, No.202 Zhenzhong Road -
West Lake Economy & TechnologyZone 310030 Hangzhou China

 SHANGHAI INTERNATIONAL TRADING CORP. GMBH Eiffestrasse 80, 20537 Hamburg, Germany

Distribuito da / Distributed by:

MORETTI S.P.A.

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE in P.R.C.

Distribuito da / Distributed by:
Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200
www.morettispa.com
info@morettispa.com