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Besmed Health Business Corp. Tapei, Taiwan

EC REP

Mdi Europa Gmbh - Langenhagener Str.71, 30855 Hannover - Langenhagen, Germany

Distribuito da / Distributed by:

MORETTI S.P.A.

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE in CHINA

II Class Medical Device according to ECC directive 93/42

- RN430** Neonatal face mask w/15mm OD
- RN431** Infant face mask w/15 mm OD
- RN432** Small child face mask w/22mm ID
- RN433** Child face mask w/22 mm ID
- RN434** Adult and large child face mask w/22mm ID
- RN435** Adult face mask w/22 mm ID

CAUTION

1. Patient should be monitored constantly whenever this device is in use
2. Store in clean dry conditions. Away from heat and light

DIRECTION FOR USE

1. Check for clear airflow passage prior to use
2. Connect to breathing circuit or resuscitation device with appropriate fitting
3. Make sure all connections are compatible and fit firmly
4. Hand-hold or secure with head strap

CLEANING AND STERILIZATION

Sterilize the components using one of the following methods:

- a. Autoclaved (max. 134° C/ 237°F)
- b. Ethylene oxide
- c. Most common disinfecting solutions which are used for equipment coming into contact with the patient. Rinse thoroughly with water after sterilization.

WARRANTY

This product has 2-year warranty from purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals. In case you need to get the product repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

Besmed Health Business Corp. Tapei, Taiwan

EC	REP
----	-----

 Mdi Europa GmbH - Langenhagener Str.71, 30855 Hannover - Langenhagen, Germany

Distribuito da / Distributed by:

MORETTI S.P.A.

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE in CHINA