## GARANZIA

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## WARRANTY

This folding walker has **2-YEAR** warranty from the purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. **The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals.** In case you need to get your walker repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

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**FOLDING WALKER WITH BRAKE AND SEAT  
INSTRUCTION MANUAL**

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I Class Medical Device  
according to ECC directive 93/42

### Code RP751

Thank you for purchasing walker **MOPEDIA** by Moretti manufactured to support people with movement difficulty. It is realised in lightweight anodized aluminium tubing with padded handles, non-slip rubber tips and casters. It is foldable and height-adjustable every 2,5 cm:

• **RP751** ..... from **82 to 92 cm**

**To adjust the height:** push the button to adjust the height. Draw out the stick paying attention to the corresponding holes up to the desired height which should reflect user's needs. Then leave the push button to fix the stick again. Repeat described procedure for each leg, at the end, check that all four legs have the same height. **Maximum endurable weight 130 kg.**

**ATTENTION:** use extra caution on wet and slippery surfaces, attention beware of electrical cords in way, wet surfaces, rugs, toys or debris, furniture, little steps and gaps in floors.  
Do not use the walker on stairs or escalators.

Avoid to leave the walker in contact with water for long periods and control its sturdiness periodically. Keep it away from children to avoid any possible risks.

With your arm extended down, handgrips should be at your wrist height.

With your hand placed on the handgrip, your elbow should be flexed by 20° - 30°.

1. To transfer the patient from the wheelchair to the walker, be sure wheels of the wheelchair are locked.
2. To walk with the walker, lift it up and place it comfortably in front of you, making sure legs end up on the floor. Walk carefully forward, one step at a time.

**Use:** aid for people with movement difficulty

**Warning:** check and secure all parts were mounted well before the use of the walker. Help the patient, if necessary, during the movement with the walker.

**Brakes:** to set off the brakes, pull up the levers placed in the lateral bar of the walker (picture 1). If you want to block the rear wheels, pull the levers down (picture 2).

picture 1/2

picture 3

picture 4

picture 5

**Front wheels:** the front castors can be both fixed and swivel, to set off the swivel function pull up the unlocking lever placed on the top of the castor (picture 3). To set off the fixed function, pull down the unlocking lever placed on the top of the castor (picture 4).

**To close the walker:** to close the walker, lift the seat and push the light blue levers alternatively, closing the lateral side (picture 5).

**Maintenance:** the walker does not need any particular maintenance, we suggest you to check periodically the rubber tips wear and the fitting of each part. Replace rubber tips every time they are worn out.

**Cleaning:** to clean the walker, use a damp cloth and a normal detergent or alcohol.

**Warning:** Moretti Spa decline all responsibility for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.

**Disposal:** the product must not be disposed of as urban waste. It must be taken a special local authority separate waste collection.