GARANZIA

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WARRANTY

This shoulder rest walker has **2-year warranty** from purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. **The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals.** In the case you need to get your shoulder rest walker repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

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SPARE PARTS

- **RV7076** Rubber wheels for walkers-pairs
- **RV7077** Rubber wheels for walkers with brakes-pairs
- **RV7090** Spare cover for antibrachium rests-pairs

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Acquistato in data / Purchasing date _____

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SHOULDER WALKER with brakes INSTRUCTION MANUAL

Moretti S.p.A.
 Via Bruxelles 3 - Melegnano
 52022 Cavriglia (Arezzo)
 Telefono +39 055 96 21 11

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• RP757 optional

ENGLISH

I Class Medical Device
according to ECC directive 93/42

Code RP755F

Thank you for purchasing demountable shoulder rest walker **MOPEDIA** by Moretti manufactured to support people with movement difficulty. It is realised in chrome plated steel tubing with four swivel wheels two of which with brakes, with shoulder rest adjustable height, width and depth.

• **RP755F** _____ fixed

How to assemble:

1. Start assembling your walker with the pole of the back frame which has to be inserted in the tubes of the lateral frame and fixed with two brass buttons.
2. Use the two horizontal poles to install in the back frame to strengthen the whole walker. Fix the poles with the four screw-buttons.
3. Adjust the rests to the patient's desired height and fix them with the two lock stems.
4. It is easy to mount and dismount the seat which is hung by means of hooks on the poles of the lateral frame (RP757 optional)

ATTENTION: Beware of electrical cords in way, wet surfaces, rugs, toys or debris, furniture, little steps and gaps in floors. Do not use the walker on stairs or escalators.

ATTENTION: USE EXTRA CAUTION ON WET AND SLIPPERY SURFACES

Avoid to leave the walker in contact with water for long periods and control its sturdiness periodically. Keep it away from children to avoid any possible risks.

• RP757 optional