GARANZIA

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WARRANTY

This shower bench has **2-YEAR** warranty from the purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. **The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals.** In case you need to get your bench repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

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Acquistato in data / Purchasing date _____

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I class medical device
according to ECC directive 93/42

Code RS915

Thank you for purchasing shower bench **MOPEDIA** by Moretti, which have been manufactured according to the highest quality and safety standards. Aluminium anodized frame with polyethylene seat foldable, rubber tips.

Place the shower bench on the floor and open it. Pay attention to position it on regular surface and check its stability before using. Maximum loading weight 100 Kg

ATTENTION: USE EXTRA CAUTION ON WET AND SLIPPERY SURFACES

Cleaning

Use a damp cloth and a normal detergent. We recommend not to submerge the seat in the bath water, drain the seat tubes after use and dry the seat thoroughly. Don't use harsh abrasives or bleach based cleaners which may cause damage or discoloration.

Warning

The manufacturer and distributor decline all responsibility for any consequences resulting from an incorrect use of this product.

- Check that the product has been delivered in perfect condition before you use it.
- If you have any doubts about the use, adjustment or maintenance of the product, please contact your dealer.
- Worn, damaged or missing components must be replaced.
- Do not store the product outdoors.
- Periodically check the stability of the product and that the screws are properly tightened and the rubber caps ends in good conditions.