## GARANZIA

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## WARRANTY

This transfer bath bench has **2-YEAR** warranty from purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. **The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals.** In case you need to get your transfer bath bench repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

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**TRANSFER BENCH  
INSTRUCTION MANUAL**

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I Class Medical Device  
according to ECC directive 93/42

## Codes RS934 - RS935

Thank you for purchasing transfer bath bench **MOPEDIA** by Moretti, manufactured for your safety. It is comfortable and easily-handled because of it's anodised aluminium structure. It is supplied with safe suction tips. The bath bench is height adjustable every 2,5cm:

- **RS935** ..... da 42 a 55 cm
- **RS934** ..... da 41 a 54 cm

### Mounting instructions

Attach back support by sliding mounting tubes through cylinder brackets and lock into place with spring loaded push pins. Back support is reversible to accommodate left or right tub entry. Attach U-shape armrest support tubing by sliding into mounting bracket and lock into place with loaded pushpins. Place legs with suction tips inside tubes RS935. Maximum endurable weight 150kg; RS934 - 100kg.

### To adjust height

Determine on which side of the unit you want to install the backrest. Make sure the unit will be facing the faucets with the bathtub. Adjust the unit to the desired height by pressing in the leg the push-button and sliding the leg upward or downward until the button snaps into the desired hole. It's possible that the legs inside the bathtub may be adjusted to a height different from the legs outside the bathtub. Place the transfer bench with the suction tips on the legs over the bathtub wall. The inner legs should be as close to the far wall of the tub as possible. Push down gently on the transfer bench to securely affix the suction tips to the bottom of the tub. To remove the unit from the bathtub, always release the suction tips by first pinch the tabs so as to break the suction.

### ATTENTION

**Make sure all the push buttons are engaged in selected holes. Before using the transfer bench, always test for stability by vigorously attempting to break the suction.**

### Maintenance

Verify regularly that all bolts and screws are properly tightened. Clean the pads with soap and water or mild liquid cleaners. Do not use harsh cleaning solvents, such as acetone which may attack the plastic parts.