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E-MAIL: info@morettispa.com

MADE IN P.R.C.

I Class Medical Device
according to ECC directive 93/42

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INTRODUCTION

Thank you for purchasing a tourniquet Easyred by Moretti S.p.A. Its design and quality guarantees comfort, safety and reliability to the user. Easyred by Moretti S.p.A. tourniquets have been designed to satisfy all your demands. This instruction manual contains advices for your safety and for a correct use of the device. In case of doubts please contact your retailer.

INTENDED USE

The tourniquet is a device that blocks the blood flow. It can be used during blood sampling or for the treatment of external hemorrhage in first aid.

EC DECLARATION OF CONFORMITY

Moretti S.p.A. declares under sole responsibility that the products made and traded by Moretti S.p.A. and belonging to tourniquet range are in accordance with the European Directive on Medical devices (93/42/EEC).

Moretti S.p.A. guarantees and declares under sole responsibility as below:

- 1) The above mentioned products satisfy all the requirements as per annex I of the directive 93/42/CEE as per annex VII° of the before mentioned directive.
- 2) The above mentioned products are NOT MEASURING DEVICES.
- 3) The above mentioned products are NOT INTENDED FOR USE IN CLINICAL VALIDATION.
- 4) The above mentioned products are NON STERILE.
- 5) The above mentioned products are to be considered "Class I° Medical Devices.
- 6) IT IS NOT ALLOWED to use the above mentioned products except for the use indicated by the Moretti S.p.A.
- 7) The manufacturer makes all the technical documentations available to the relevant authorities which prove the compliance with the 93/42/EEC Directive.

HOW TO USE THE TOURNIQUETS

1. Push the button on the buckle and open it;
2. Put the tourniquet around the arm and close the buckle;
3. Pull the elastic band outward until the arm is blocked;
4. Push again the button on the buckle to open the tourniquet.

WARNING!

Read carefully this instruction manual for a correct use. Pay attention to the presence of children.

CLEANING

Use warm water and neutral soap to clean the product.

GENERAL WARNINGS

Do not use the product for a purpose not indicated in this manual. Moretti S.p.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration of the product. The manufacturer can modify the device or the instruction manual without prior notification.

TECHNICAL FEATURES

Material: Nylon covered latex
Length: 45 cm
Width: 2,5 cm
Weight: 32 g

GARANZIA

The product has 2-year warranty from purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals. In case you need to get the product repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

WARRANTY CERTIFICATE

Product _____

Purchasing date _____

Authorized dealer _____

Street _____ Place _____

Sold to _____

Street _____ Place _____

Moretti S.p.a.

INTERNET: www.morettispa.com

E-MAIL: info@morettispa.com

MADE in P.R.C.