**KYARA**

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**MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

**www.morettispa.com email: info@morettispa.com****MADE IN ITALY**

**INDEX**

CODE .....	Pag.11
INTRODUCTION.....	Pag.11
INTENDED USE .....	Pag.11
GENERAL WARNING .....	Pag.11
GENERAL DESCRIPTION.....	Pag.11
- Parts view .....	Pag.11
USING WARNINGS .....	Pag.12
HOW TO USE.....	Pag.12
MAINTENANCE .....	Pag.13
CLEANING AND DISINFECTION.....	Pag.13
CONDITIONS OF DISPOSAL.....	Pag.13
- General conditions of disposal.....	Pag.13
- Treatment of electrical parts (Directive 2002/96/EC) .....	Pag.13
SPARE PARTS/ ACCESSORIES .....	Pag.14
TECHNICAL FEATURES.....	Pag.14
SYMBOLOLOGY .....	Pag.14
WARRANTY .....	Pag.15

**CODE**
**LT170** Room humidifier "BUNNY"

**INTRODUCTION**

Thank you for purchasing the KYARA humidifier by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

**INTENDED USE**

The humidifier is intended to maintain a correct humidity level in dry rooms.

**CAUTION!**


Do not use the product for a purpose not indicated in this manual. Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product. The manufacturer reserves the right to change the information contained in this document without previous notice.

**GENERAL WARNINGS**

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual  
If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment, otherwise injury or damage may occur
- Keep the packed humidifier away from heat sources
- Avoid direct exposure of the device to direct sunlight or other heat source
- SERVICE LIFE - the humidifier use limit is defined by the wear of the parts
- DO NOT allow children to play on or operate the humidifier

**GENERAL DESCRIPTION**

Parts view

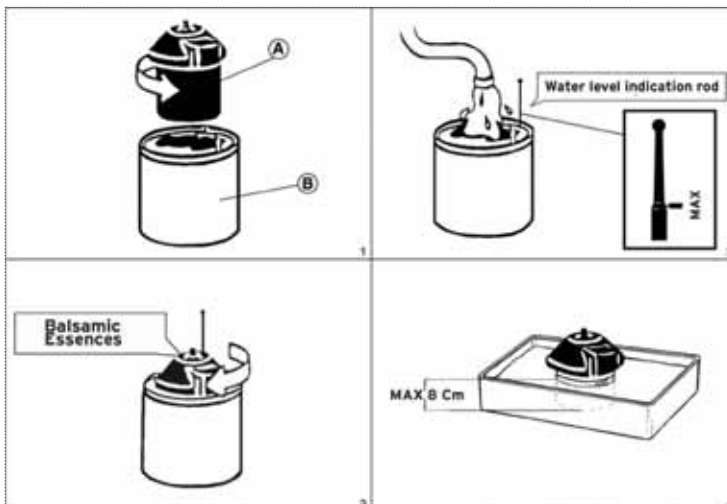- A.** Vapour generator
- B.** Water container

### USING WARNINGS

- If the appliance casing or the power supply cable are damaged, do not use the appliance
- Do not leave any of the packaging (plastic box, cardboard boxes, etc.) within the reach of children or disabled persons are potential sources of danger
- Only use the appliance for its intended use- “room humidifier”- and as described in this manual
- The appliance has started protection against the entry of liquids
- The appliance is not suitable for use in the presence of an inflammable anaesthetic mixture with air, oxygen or nitrous oxide
- Do not use the appliance with wet or damp hands, or when taking bath
- Do not wet the upper part of the vapour generator because it contains electrical parts. If it gets wet accidentally, contact qualified technical personnel to check the appliance
- If necessary, use adapters or extension leads that comply to the safety standard in force, paying attention not to exceed the rated current or maximum values shown on the adapter
- Only use the vapour generator with the container (tank) supplied
- Keep the appliance and the power supply cable far away from sources of heat
- Position the appliance out of the reach of children
- Do not position the appliance on a soft surface (like bed or a divan)
- When the appliance is working its surface warm up. Do not locate it on places sensitive to heat (like glass, treaty materials, etc.)
- Steam is naturally very hot and therefore to avoid the risk of scalds, avoid directing the stem jet towards people, animals, objects, articles and walls
- Do not let the children or disabled persons use the appliance as a toy
- Do not overturn, move or handle the appliance when operating  
Any spillage of boiling water could cause scalds and electric shocks
- Only use balsamic essences or anti-lime scale tablets certified, to be “non-toxic”
- Do not block the openings in the vapour generator; do not insert any objects into the openings
- Switch off the appliance and remove the plug from the power socket, before extracting the vapour generator
- Do not pull the power supply cable or the appliance to remove the plug from socket
- Always remove the plug after use
- Before handling, cleaning or filling, wait the appliance has cooled down sufficiently
- Do not leave the appliance exposed to the weather
- Do not supply the appliance with direct current
- During operation place the appliance in a stable position so as to avoid impacts or accidental falls of the same, possibly causing the spillage of boiling water or malfunctioning

### HOW TO USE

1. Keeping the appliance disconnected from the power socket, turn the steam generator anticlockwise to make the arrow line up with the “O” symbol. Next, pull it out of the container
2. Pour cold water into the container until the swelling on the level indication rod appears. Use only tap water and under no circumstances introduce other liquids or substances. **Do not use demineralised water because it damages the**

**humidifier's performance.** Do not use salted water or water belonging to purifiers using salt

3. Rest the appliance on a stable surface out of the reach of children and insert the vapour generator with the arrow lined up with the "O" symbol. Insert the plug into the power supply socket and turn the vapour generator clockwise to make the arrow line up with the "I" symbol. In this way, the switch works automatically **Bear in mind that a few minutes are necessary before vapour production starts. Do not shift or move the appliance during operation**
4. To vaporise and disperse balsamic essences in the room, pour a few drops of the same into the small vessel located under the vapour delivery hole. **Do not pour balsamic essences into the water container as the appliance could be irreversibly damaged**
5. When the water in the container runs out, vapour production ceases automatically. Remove the plug from supply socket as soon as possible
6. Before introducing water into the container or before putting it away, we recommend switching it off, removing the plug from the socket and waiting until it has completely cooled down, next, empty any residual water from the container and rinse it carefully

## MAINTENANCE

The KYARA devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. In case of repair, use only original spare parts and accessories.

## CLEANING AND DISINFECTION

- All the maintenance operations must be performed with the appliance plug removed from the power supply In case of disinfection use only mild soap and water
- Calcium present in the water deposited as lime-scale on the appliance electrodes can cause a reduction in the vapour produced
- Periodically clean the vapour generator electrodes by inserting it into a vessel with a solution of water and anti-lime or vinegar and let this work for a few hours
- Do not submerge the vapour generator in the solution beyond the level max. 8 cm before using the vapour generator repeat the previous operation a few times, only using water

## CONDITIONS OF DISPOSAL

### General conditions of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the humidifier in the appropriate disposal areas for recycling.

## TREATMENT OF ELECTRICAL PARTS

(Directive 2002/96/EC)

At the end of its life, the product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment or at retailers that provide this service. By ensuring these batteries are disposed of correctly, you will help prevent potentially negative consequences for the environment and human health which could otherwise be caused by inappropriate waste handling of the battery. To remark the need to dispose of electrical equipment separately, the products is marked with crossed mobile waste bin

## SPARE PARTS/ACCESSORIES

For spare parts and accessories refer to Moretti main catalogue

## TECHNICAL FEATURES

Water heating:	<b>by electrodes</b>
Power supply:	<b>230V 50/60Hz*</b>
Starting input:	<b>max 800W*</b>
Input:	<b>250W*</b>
Steaming time:	<b>5/6 hours*</b>
	<b>2 liter tanks</b>
Weight:	<b>1,1kg</b>
Dimension:	<b>180 mm diameter</b>
	<b>190 mm height</b>

\* (variable data in function of the conductivity of the water)

**NOTE** a safety fuse is housed in the appliance which can be replaced if it blows. As for all other possible repairs, the replacement operation must be performed by the producer company or by personnel authorized by the same. Before replacing the fuse, remove the plug from the power supply socket. Replace the fuse with one with the same electrical characteristic: **T6, 3A 250V**

## SYMBOLOLOGY

Read the instruction carefully

Class II

0 ON

Conforms to the European directives  
2006/95/EC

I OFF

Manufacturer for 2006/95/EC  
Directive

Caution- Hot steam

Disposal

Recycled or recyclable  
packaging

Recycling contribution

Lot

Serial number

## **WARRANTY**

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows.

The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual.

Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

## **REPAIRING**

### **- Warranty Repair -**

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters.

Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

### **- Repair not covered by warranty -**

A product out of warranty can be sent after Moretti authorization.

The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

### **- Non-Defective device -**

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be paid by the customer.

## **SPARE PARTS**

The original Moretti spare parts are guaranteed for 6 months from the receiving date

**EXEMPT CLAUSES**

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

**WARRANTY CERTIFICATE**

**Product** \_\_\_\_\_

**Purchased on (date)** \_\_\_\_\_

**Retailer** \_\_\_\_\_

**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_

**Sold to** \_\_\_\_\_

**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_

**MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

[www.morettispa.com](http://www.morettispa.com) email: [info@morettispa.com](mailto:info@morettispa.com)

**MADE IN ITALY**