## BILANCIA PESANEONATI BABY SCALE

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**CODE**
**LTK630** Baby scale

**INTRODUCTION**

Thank you for purchasing the **KYARA** baby scale by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

**INTENDED USE**

This scale is intended to detect babies weight.

**CAUTION!**

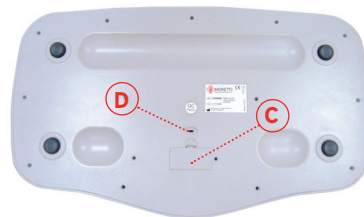
- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

**GENERAL WARNINGS**

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment otherwise injury or damage may occur
- Advise with physician or therapist to determinate the correct adjustment and the correct use of the scale
- Keep the packed scale away from heat sources
- SERVICE LIFE- the scale use limit is defined by the wear of the parts
- Protect the scale from impact with hard objects, moisture, dust, chemicals, liquid cosmetics and great temperature fluctuation
- Do not place any objects on the scale when not in use
- Strong electromagnetic fields may impair the accuracy of the scale
- Remove the battery if the scale is not going to be used for a long period of time

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## ASSEMBLING

### Battery assembling

These kind of scale do not require special assembly operations, excluding the installation of the battery that must be completed in the type and model stated in this manual and the polarity marked on the battery compartment.

## BEFORE USE

Place the scale on a level and solid surface in order to ensure the scale weighs correctly

## HOW TO USE

1. Install no.1 3V CR2032 lithium battery in the battery compartment, matching the indicated polarity
2. Press the button (⏻) to switch the scale
3. Select the correct units on the back of the scale acting on the switch (LB=LB/ KG=KG/ ST=Stone)

### NOTE THIS OPERATION MUST BE DONE ONLY ONCE, DURING THE FIRST USAGE

4. The scale is ready to take measurement as soon as "0.000 kg" appears on the display
5. Place the baby on the weighing area
6. The scale has a "AUTO HOLD" function, which establishes the baby's weight as soon as he/she is still enough to achieve an exact measurement value
7. The word "HOLD" stops flashing and the weight is displayed for a further 20 seconds
8. If the baby is wriggling about too vigorously, press the HOLD button. Within 3 seconds the weight is displayed for a further 20 seconds
9. If is preferred to use a towel on the scale, place it before switching on the scale
10. To reset the scale to 0.000g, press TARE button after "HOLD" stopped flashing
11. When "Lo" appears on the display, the battery need to be replaced

## MAINTENANCE

The KYARA devices by Moretti S.p.A. are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years.

## CLEANING AND DISINFECTION

- **Cleaning:** use a damp cloth and mild soap. Then dry thoroughly before use. Never immerse the scale in water or rinse it under running water

- **Disinfection:** if you need to disinfect the device use a common disinfectant cleaner

### NOTE NEVER USE ACIDS, ALKALIS OR SOLVENTS SUCH AS ACETONE

## CONDITIONS OF DISPOSAL

### - General conditions of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the scale in the appropriate disposal areas for recycling.

**DISPOSAL OF BATTERIES AND ELECTRONIC COMPONENTS**

**DISPOSAL INSTRUCTIONS, AS REQUIRED BY DIRECTIVE 2002/96/EC:**

At the end of its service life, the product may not be disposed together with normal municipal waste. It must be delivered to the municipal collection disposal centre or to the reseller, who provides this service. Sorted waste disposal reduces the environmental and health consequences of improper disposal and enables its materials to be recycled, this resulting in energy and resource savings. To highlight the requirement for sorted disposal of electromedical equipment, the product bears the barred bin symbol.

**DISPOSAL OF WASTE BATTERIES  
(Directive 2006/66/Ec)**

This symbol on the battery or on the packaging indicates that the battery provided with this product shall not be treated as household waste. By ensuring these batteries are disposed of correctly, you will help prevent potentially negative consequences for the environment and human health which could otherwise be caused by inappropriate waste handling of the battery. The recycling of the materials will help to conserve natural resources. At the end of their life hand the batteries over to the applicable collection points for the recycling of waste batteries. For more detailed information about recycling of this product or battery, please contact your local Civic Office, your household waste disposal service or the shop where you purchased the product.

**TECHNICAL FEATURES**

<b>Dimensions</b>	<b>33x56 cm h-5,4cm</b>
<b>Weight max.</b>	<b>20kg</b>
<b>Graduation</b>	<b>5 gr</b>
<b>LCD display</b>	<b>2.9"</b>
<b>Power supply</b>	<b>1 x 3V CR2032 model</b>
<b>Hold function</b>	
<b>Tare function</b>	
<b>Auto-off</b>	

**WARRANTY**

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual.

Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to mistaken installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

**REPAIRING**

**Warranty Repair**

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

**Repair not covered by warranty**

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

**Non-Defective device**

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be paid by the customer.

**EXEMPT CLAUSES**

Moretti does not offer any other declarations, warranty or conditions, explicit or implicit, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

**WARRANTY CERTIFICATE**

Product \_\_\_\_\_

Purchasing date \_\_\_\_\_

Authorized dealer \_\_\_\_\_

Street \_\_\_\_\_ Place \_\_\_\_\_

Sold to \_\_\_\_\_

Street \_\_\_\_\_ Place \_\_\_\_\_

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

[www.morettispa.com](http://www.morettispa.com) email: [info@morettispa.com](mailto:info@morettispa.com)

MADE IN P.R.C.