**MOPEDIA®**

MA MP310-315 01 A\_03-2014

**PARASPONDE  
MANUALE D'ISTRUZIONE**

---

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**MP315** PMP315 Folding bed rail protector 127X66X2.5h cm

### 2. Introduction

Thank you for purchasing a Mopedia bed rail protector by Moretti S.p.A. these devices are designed and realized to meet all your needs for a practical, proper and safe use. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

### 3. Intended use

The **MOPEDIA** by Moretti bed rail protector are designed to protect the patient from possible critical accidental injury against the sides of the bed.

#### ATTENZIONE!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

### 4. EC declaration of conformity

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the cane aids category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications.

To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/ EEC as prescribed by Annex VII of the above directive
2. The devices in question **ARE NOT MEASUREMENT DEVICES**
3. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**
4. The devices in question are sold in **NON STERILE** packing
5. The devices in question should be considered as belonging to class I
6. **DO NOT** use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities and further modifications.

### 5. General warnings

- **READ** carefully the entire manual before using the device .
- For a correct use of the device always consult your physician or therapist
- Make sure that the product is delivered complete and keep it packed away from heat sources
- We recommend using a single user
- The device must be sanitized before any re-use **SERVICE LIFE** the device use limit is defined by the wear of the parts
- **DO NOT** allow children to play or operate the device

## 6. General description

The MOPEDIA by MORETTI bed rail protectors are made of memory foam (polyurethane) and complete with waterproof cover (40% PU and 60% polyester) - (removable by zip for model MP310 non-removable for model MP315.

The cover is made with fabric with the following features:

- a) impermeable to liquids,
- b) vapor permeable,
- c) two-way stretch,
- d) antibacterial, antifungal treatment Sanitized
- e) latex-free (hypoallergenic)

## 7. How to use

Fit the device with the long side facing the bed.

### MP310

Adaptable to any type of bed rail thanks to the 4 adjustable straps

### MP315

Folding model is easy to install thanks to the velcro closure

## 8. Cleaning

### MP310

Indication for washing ONLY FOR COVER:

Maximum temperature: 30 ° C

Do not bleach

Do not iron (do not use steam)

Do not dry clean, do not use stain removers with solvents

Do not tumble dry

**Do not wash the internal foam**

### MP315

Use only a damp cloth to remove dust, use a damp cloth with a mild soap to remove dirt more difficult. Dry thoroughly before use

## 9. Warnings and advises

We recommend the use of a single user.

- Store and use away from open flames and heat sources
- The devices must be sanitized prior to any use
- You are not aware of any side effects.

## 10. Technical features

MP310 Dimensions: **134 x 33 x 3h cm**

MP315 Dimensions: **127 x 66 x 2,5h cm**

Composition MP310 e MP315:

Internal material -> **PU**

COver -> **40% PU 60% Polyester**

## 11. Maintenance

The MOPEDIA devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years.

## 12.1 Condition of disposal

### 12.1 General condition of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

## 13. Warranty

Moretti products are guaranteed from material or manufacturing faults for a period of ( two ) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product , and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster , not authorized maintenance or repair , damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti ), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

### 13.1 Exempt clauses

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

**WARRANTY CERTIFICATE**

**Product** \_\_\_\_\_

**Purchased on (date)** \_\_\_\_\_

**Retailer** \_\_\_\_\_

**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_

**Sold to** \_\_\_\_\_

**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_

 **MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

[www.morettispa.com](http://www.morettispa.com) email: [info@morettispa.com](mailto:info@morettispa.com)

**MADE IN P.R.C**

Distribuito da / Distributed by:

Via Bruxelles, 3 - Meleto  
52022 Cavriglia (Arezzo)  
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200  
[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)