MORETTI

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200
www.morettispa.com
info@morettispa.com

RP755S

- Fisso
- Fixed

RP755T

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INTERNET: www.morettispa.com E-MAIL: info@morettispa.com

MADE in P.R.C.

I Class Medical Device
according to 93/42/EEC directive

Codes: 755S - RP755T - RP753S

Thank you for choosing a demountable walker **MOPEDIA** by Moretti S.p.A. product range, manufactured to support people with movement difficulty. It is realised in chrome plated steel tubing with four swivel wheels two which with brakes. The RP755S and RP753S models are equipped with padded armrests, adjustable height, width and depth. The RP755T is equipped with padded table

- **RP755S** fixed
- **RP755T** fixed with padded table
- **RP753S** adjustable height every 2,5 cm from 117 cm to 135 cm

RP755S and RP753S assembling instruction


1. Start assembling your walker with the pole of the back frame which has to be inserted in the tubes of the lateral frame and fixed with two brass buttons
2. Use the two horizontal poles to install in the back frame to strengthen the whole walker. Fix the poles with the four screw-buttons
3. Adjust the rests to the shoulder rest to the patient's desired height and fix them with the two lock system.
4. The seat can be easily assembled or removed thanks to the hooks on the poles of the lateral frame. (RP757 optional)

- RP757 optional

RP755T assembling instruction

VIEW OF THE PARTS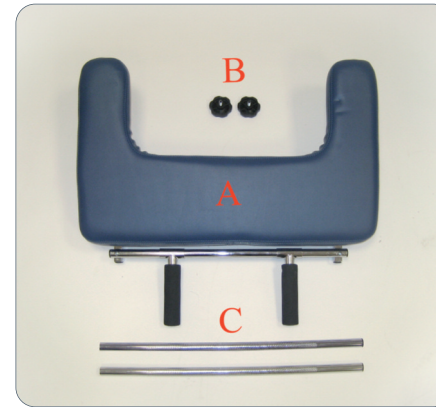

LIST OF THE PARTS

- A. Padded table with handles
- B. Handles lock knobs
- C. Table support poles (length 560mm or 720 mm depending on the model)

ASSEMBLY

1. Start assembling your walker with the pole of the back frame which has to be inserted in the tubes of the lateral frame and fixed with two brass buttons.
2. Use the two horizontal poles to install in the back frame to strengthen the whole walker. Fix the poles with the four screw-buttons.
3. To assembly the padded armrest, insert the poles (ref.C) into the support tubes, once they are at the same height block them with the levers on the walker.
4. Place the padded table (ref.A) and adjust the height, in order the user can rest his arms forming a 90° angle.
5. Adjust the handles at the desired opening and block them with the screw-buttons supplied with the padded table.
6. Assure that the screws are duly tightened.

7. The table is now correctly assembled and ready to use. As shown in fig.1

8. The seat can be easily assembled or removed thanks to the hooks on the poles of the lateral frame. (RP757 optional)

TECHINICAL FEATURES


- Steel chrome tubular frame
- Rest table composed by foam rubber
- Synthetic leather cover

- a) 41 cm
 b) 66 cm
 c) 11 cm
 d) 44 cm
 e) 21 cm
 f) 13,5 cm
 g) 6 cm
 h) 15 cm

Temple length:

- (RP759/1) = 56 cm
 (RP759/2) = 72 cm

Total weight inclusive temple length: 2 Kg

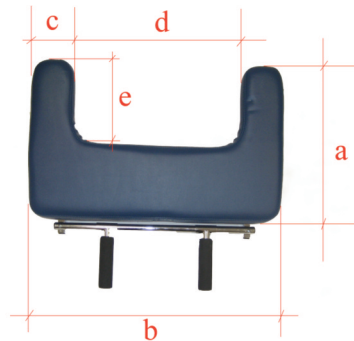WARNING

Beware of electrical cords in way, wet surface, rugs, toys or debris, furniture, little steps and gaps in floors. Do not use the walker on stairs or escalator.

USE EXTRA CAUTION ON WET AND SLIPPERY SURFACES

Avoid to leave the walker in contact with water for long periods and check its sturdiness periodically. Keep it away from children to avoid any possible risks

SPARE PARTS

- **RV7076** Rubber wheels for walkers-pairs
- **RV7077** Rubber wheels for walkers with brakes-pairs
- **RV7090** Spare cover for antibrachium rests-pairs

ACCESSORIES

- **RV7072** Padded shoulder rest-pairs
- **RV708** Antibrachium rest-pairs
- **RP7591-2** Padded table for RP755F, RP755S, RP756A e RP756S

WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be apply in the event damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual.

Moretti is not responsible of consequent damages, personal injuries or whatever caused by or in relation with mistaken installation or with device improper use.

Moretti does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance or repairs , faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

REPAIRS

Warranty repair

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will estimate with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, could repair or replace the item, with a Moretti dealer or with own base. Labor cost can be in charge to Moretti if the repair is cover from warranty. A repair or a replacement doesn't extend the warranty.

Repair not cover from warranty

A product without warranty can be given back after Moretti authorization. The labor and shipping costs for a bath lift out of warranty are to be payed by the customer or by the dealer. The repairs are guaranteed for 6 months as from the good received.

Non-defective device

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be payed by the customer.

SPARE PARTS

The original Moretti spare parts are guaranteed for 6 months as from the receiving date.

EXEMPT CLAUSES

Moretti doesn't offer any other declarations, warranty or conditions, explicit or implicit, including possible declarations, warranties or condition of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti doesn't guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be fixed by according to the law is limited by the warranty period, in the limits of law. Some states or countries don't allow to limit the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

WARRANTY CERTIFICATE

Product _____

Purchasing date _____

Authorized dealer _____

Street _____ Place _____

Purchased by _____

Street _____ Place _____

MORETTI S.p.A.

INTERNET: www.morettispa.com E-MAIL: info@morettispa.com

MADE in P.R.C.