MOPEDIA

• RS941

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I Class Medical Device
according to ECC directive 93/42

Codice RS941

Thank you for purchasing a shower chair **MOPEDIA** by Moretti. Its design and its quality guarantee to the user comfort, safety and reliability. The commode chairs and the shower chairs **MOPEDIA** by Moretti are designed and realized to meet your requirements for a correct use in inner spaces. The product purchased has been studied and analyzed in Italy from the research and development department of Moretti S.p.A. Italia. The material used, the accessories and the quality test are continuously checked by Moretti S.p.A. staff. Only the production is made in Popular Republic of China according to the highest quality and safety standards in accordance with the European Directive on Medical devices (93/42/CEE).

This instruction manual give you suggestion for a correct use of the aid and good safety advice. We recommend to read carefully this instruction manual before using the chair. In case of doubts about the use, please contact your dealer.

Use

The shower chair **MOPEDIA** is created for people with movement difficulty and can be used indoors as substitution of toilet or as aid under the shower. Moretti Spa decline all responsibility for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.

CE Declaration of conformity

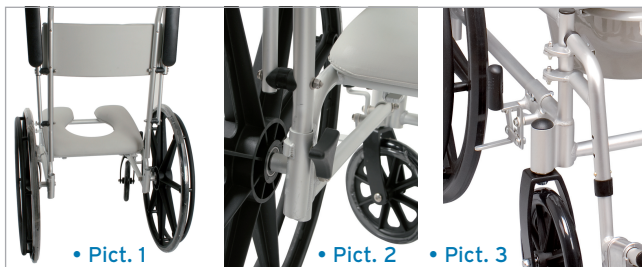
Moretti S.p.A. declare under sole responsibility that the product made and traded by Moretti S.p.A. and belonging to **Shower chair** range are in accordance with the European Directive on Medical devices (93/42/CEE).

Moretti S.p.A. guarantee and declared under sole responsibility what follow:

1. The devices under discussion satisfy the essential requisites requested by the I° enclosure directive 93/42/CEE as laid down by the VII° enclosure of the above mentioned Directive.
2. The devices under discussion **ARE NOT MEASURING INSTRUMENT.**
3. The devices under discussion **ARE NOT MADE FOR CLINICAL PROBES.**
4. The devices under discussion are traded with **NON STERILE BOX.**
5. The devices under discussion belong to class I°.
6. **IT IS NOT ALLOWED** to use and install devices except for the use provided by Moretti S.p.A. Moretti S.p.A. places technical documentation at relevant authorities in order to prove the conformity of 92/42/CEE device.

Assembly instruction

All the Mopedia Chairs are delivered with disassembled backrest and footrest.

To assemble the seat insert the tubes in the opening on the back of the seat (Picture 1).

The adjustable footrest have to be fixed on the external side of the seat with the pins (Picture 2). Pay attention to the right footrest to use and note that the footrest have to be fixed inwards. To fix the footrest be sure that the locking

lever is well fixed inwards the chair. Insert the WC from the back part of the chair using the rails (Picture 3).

Footrest adjustment

The footrest of shower chair can be adjusted according to various requirements. You have to push the locking clip on the footrest (Picture 4) not less than 6.5cm distant from the floor. Check that both footrest are adjusted at the same height.

• Pict. 4

Use

Use only the back handle to drive the chair, do not use other supports. Know your own capabilities and limitations in terms of strength and endurance before attempting to go up a ramp. You and your attendant should first consult a qualified professional for correct techniques on ramp negotiation. Assistance is recommended when going up or down step inclines greater than 10%. If it becomes necessary to stop on an incline, avoid sudden, abrupt forward movement when you resume propelling the chair. If you want to lift the chair up, hold it from back handles and frame only.

WARNING

The patient must not push the footrest when go up and down from the chair.

Brakes

Engage wheel locks on both wheels before entering the chair, leaving the chair, leaning forward in the chair or while being transported in an elevator or wheelchair lift. On models with small wheels, push the lever on each wheel; (Picture 5) on the other models lift up the lever to brake. The lever on the brake must tighten the wheel. In case of necessity check the pressure of the casters. If necessary adjust the loosen nut and then tighten up the nut.

• Pict. 5

Castors

Castors should be inspected periodically and lubricated when necessary. If the wheel wobbles, tighten the nut of the pivot slightly. To test for correct tightness, lean the chair to one side and spin the adjusted wheel.

Armrest

To facilitate the transport of the wheelchair you can remove or overturn the armrests. Never use them to move or to lift the chair.

WC

Use the toilet when the wheelchair is still. To use the toilet take out the removable part of the seat using the handle on the front part. Take out the toilet using the handle ([Picture 6](#)).

• Pict. 6

Measures wheelchair

Warning

- Don't attempt inclines greater than 10% without assistance.
- Completely engage the wheels locks when entering and leaving wheelchair or using it in elevator or wheelchair lift.
- Wheelchair should be on a stable level surface and wheels locks should be engaged before transferring user to or from the wheelchair.
- Before leaning or reaching forward, rotate the casters toward the front of the wheelchair and engage wheel locks to help prevent wheelchair from tipping over.
- Do not lean over the top of the wheelchair back since this can cause the wheelchair to tip over.
- Pay attention that the clothes does not get entangled in the wheels of the chair.
- Pay attention to avoid the crushing of fingers during the assembling or the regulation of footrest, backrest or wheels and during the use of the chair.
- Keep the chair far from fire.

Maintenance:

The chair **MOPEDIA** by MORETTI require minimal maintenance and general care, you have to control regularly:

- Wheels locks: *monthly*.
- Cleaning of the seat: *when necessary*.
- Professional inspection: *biennial*.
- For the clearing of chromate or metallic parts of the seat use water and soap used for cars. Seat and backrest can be cleaned with a damp cloth and a normal detergent. The toilet must be clean and disinfected regularly using a normal bath detergent. Clean always before the use.

Disposal: the product must not be disposed of as urban waste. It must be taken a special local authority separate waste collection.

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WARRANTY

This shower chair has **2-YEAR** warranty from the purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. **The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals.** In case you need to get this product repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

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Acquistato in data / Purchasing date _____

Rivenditore / Authorized dealer _____

Via / Street _____

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Venduto a / Purchased by _____

Via / Street _____

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MORETTI S.p.A.

INTERNET: www.morettispa.com E-MAIL: info@morettispa.com

MADE in P.R.C.