## Codici

ST587 Dispositivo ginnastica respiratoria 2,5 litri

ST588 Dispositivo ginnastica respiratoria 5 litri

## Codes

ST587 Volume oriented respiration exerciser 2,5 l

ST588 Volume oriented respiration exerciser 5 l

EN

## PRODUCT DESCRIPTION

RESPIADVANCE is a volume oriented single patient respiration exerciser designed to encourage Sustained Maximal Inspiration in patients with respiratory difficulties. The repeated Inspiratory Muscle Training (IMT) helps to improve the strength of respiratory muscles providing a progress in the respiratory functional parameters. RESPIADVANCE consents the visible perception of the progress.

## INDICATIONS

To prevent postoperative respiratory complications, For patients prescribed to perform Respiratory Muscle Training therapy, like patients showing malfunction of the respiratory muscles in illness that predispose to respiratory insufficiency as COPD (Chronicle Obstructive Pulmonary Disease).

## CONTRAINDICATIONS

The RESPIADVANCE is contraindicated for patients known to have pneumothorax. Contact the doctor or therapist for further information.

## Contents of the bag

A) RESPIADVANCE B) Connector with particle filter; C) Tube with mouthpiece; D) Detail of the connector, the filter and the tube.

## DIRECTIONS FOR USE

RESPIADVANCE must always be used following the directions and instructions of a doctor or therapist. Remove the device from package and connect the tubing to the orifice with filter located in the centre of the device. (fig. 1). The goal to achieve during exercise must be set in by the therapist, by means of the upper mobile indicator located in the right column, or marking it with ink on the surface. Seat comfortably to promote optimal lung expansion. If you are unable to assume or maintain this position, perform the procedure in any position as long as the device remains upright. Tilting the RESPIADVANCE decreases the required effort and reduces the exercise effectiveness.

- 1) Breathe normally and place lips tightly around mouthpiece. (fig 2)
- 2) INHALE slowly till the mobile float in the left column (flow indicator) is placed in BETWEEN the transparent window area. While inhaling, maintain it floating in between the window as long as possible (fig 3). You will perceive that the right mobile float start to rise. Continue inhaling, trying to reach the target marked by the therapist.
- 3) Remove the mouthpiece from lips. EXHALE slowly through the nose, while both orange mobile floats return to bottom of columns.
- 4) Use the lower mobile indicator to mark up in the right column the level reached. (fig 4)
- 5) Relax after each deep breath.
- 6) Repeat the process at the frequency prescribed by your therapist.
- 7) Cough after the exercise to remove accumulated secretions.
- 8) Place the tube with the mouthpiece in its seat lid top when the exercise is finished.(fig 5)
- 9) Record the level reached in your daily progress chart to show to your physician.

## CAUTION.

Frequency of use and target inspiratory values must be settled by your physician or therapist. In paediatric use, the child must be supervised during exercise. As any medical device, keep it out of the reach of children.

## CLEANING THE RESPIADVANCE

After each use, wash carefully the mouthpiece with warm (approx 40°C) spin water, and leave it dry. Once a week, wash the three filters located at the bottom of the device (fig 6). When not in use, keep the device in a dry clean place. The device is latex free and may be disposed in the plastic clearance container. NOTE to physicians and healthcare professionals: The RESPIADVANCE is not a volume measuring device. The figures shown and given are only approximated. The filter located in the insertion point of the tubing serves only to prevent the inhalation of particles.

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